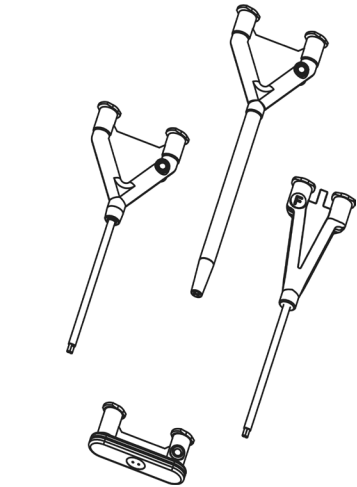
Instructions for Use  
Mode d'emploi  
Gebrauchsanweisung  
Instrucciones de uso  
Istruzioni per l'uso  
Gebruiksaanwijzing  
Bruksanvisning  
Instruções de Utilização  
Kullanma Talimatı  
Инструкция по эксплуатации

MANUFACTURED FOR:

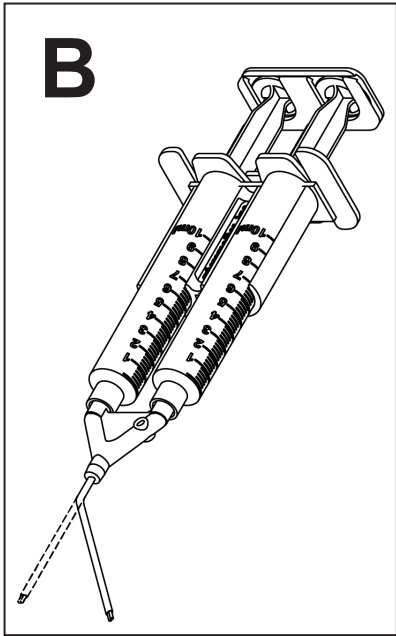
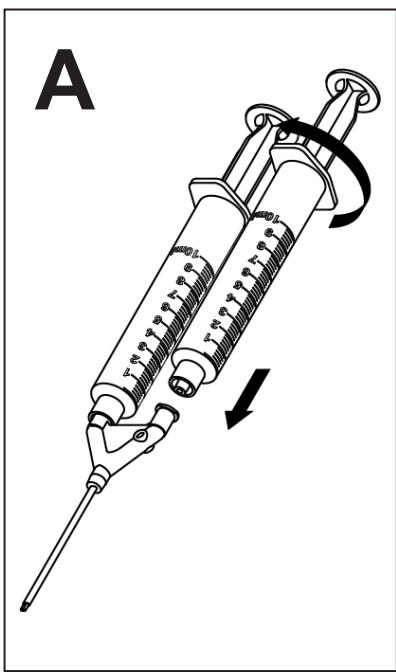
  
Micromedics, Inc.  
1270 Eagan Industrial Road  
St. Paul MN 55121-1385, USA  
651-452-1977  
800-624-5662  
info@nordsonmedical.com  
www.nordsonmedical.com

CE 0297  
EC REP  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover, Germany

| Symbol Reference Key /<br>Définition des symboles /<br>Erklärung der Symbole /<br>Clave de referencia a los símbolos /<br>Legenda dei simboli /<br>Verklaring van symbolen /<br>Nøkkel til symbolreferanser /<br>Legenda dos Símbolos /<br>Simge Başvuru Anahtarı /<br>Условные обозначения                                                                                                                                                                                                                                                                   |                                                                                                    |                                                                                                 |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------|
| APPLICATOR TIP                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                |                                                                                                    |                                                                                                 |
| Applicator Tip<br>Embout de pulvérisateur<br>Applikatorspitze<br>Punta aplicadora<br>Punta per applicatore                                                                                                                                                                                                                                                                                                                                                                                                                                                    | Applikatorspiss<br>Ponta Aplicadora<br>Applikator Ucu<br>Наконечник аппликатора                    |                                                                                                 |
| APPLICATOR TIP DUAL CANNULA, MALLEABLE                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                        |                                                                                                    |                                                                                                 |
| Applicator Tip dual cannula, malleable<br>Canule double à Embout de pulvérisateur, malléable<br>Applikatorspitze mit doppelkanüle, formbar<br>Punta aplicadora con doble cánula, maleable<br>Punta per applicatore, cannula doppia, malleabile<br>Dubbele canule applicatorip, vormbaar<br>Applikatorspiss med dobbel kanyle, formbar<br>Cânula dupla de Ponta Aplicadora, maleável<br>Applikator ucu çift kanüllü, şekillendirilebilir<br>Наконечник аппликатора с двойной канюлей, пластичный                                                               |                                                                                                    |                                                                                                 |
| APPLICATOR TIP, DUAL CHANNEL FLEXIBLE CATHETER                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                |                                                                                                    |                                                                                                 |
| Applicator Tip, dual channel flexible catheter<br>Embout de pulvérisateur, cathéter flexible à double canal<br>Applikatorspitze, doppelkanal und biegsamer katheter<br>Punta aplicadora, catéter flexible de canal doble<br>Punta per applicatore, catetere flessibile a doppio canale<br>Applikatorip, buigzame katheter met twee kalen<br>Applikatorspiss, fleksibelt kateter med dobbel kanal<br>Ponta Aplicadora, cateter flexível de canal duplo<br>Applikator Ucu, çift kanallı esnek kateter<br>Наюнечник аппликатора с двухканальным гибким катетером |                                                                                                    |                                                                                                 |
| APPLICATOR TIP, DUAL LUMEN ENDOSCOPIC                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |                                                                                                    |                                                                                                 |
| Applicator tip, dual lumen endoscopic<br>Embout de pulvérisation, endoscopique à double lumière<br>Applikatorspitze, doppellumig für die endoskopie<br>Punta aplicadora, endoscópica, de doble lumen<br>Punta per applicatore, catetere endoscopico a doppio lume<br>Applikatorip, endoscopische, met twee lumina<br>Applikatorspiss, endoskopisk med to løp<br>Ponta aplicadora, endoscópica de duplo lúmen<br>Applikator ucu, çift kanüllü endoskopik<br>Наконечник аппликатора двухпросветный эндоскопический                                              |                                                                                                    |                                                                                                 |
| BLENDING CONNECTOR WITH MIXER                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                 |                                                                                                    |                                                                                                 |
| Blending connector with mixer<br>Connecteur de mélange avec mélangeur<br>Mischanschluss mit mischer<br>Conector para mezcla con mezclador<br>Connettore di miscelazione con mixer<br>Mengconnector met mixer<br>Blandekobling med mikser<br>Combinação do conector com o misturador<br>Mikserli karıştırıcı konektör<br>Коннектор для смешивания, со смесителем                                                                                                                                                                                               |                                                                                                    |                                                                                                 |
| MANUFACTURED FOR:                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             |                                                                                                    |                                                                                                 |
| Manufactured for:<br>Fabriqué pour :<br>Hergestellt für:<br>Fabricado para:<br>Prodotto per:<br>Vervaardigd voor:<br>Produsert for:<br>Fabricado para:<br>Hangi firma için üretildiği:<br>Изготовлено для:                                                                                                                                                                                                                                                                                                                                                    |                                                                                                    |                                                                                                 |
| REF                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                           | Catalog Number<br>Numéro de référence<br>Katalognummer<br>Número de catálogo<br>Número de catálogo | Catalogusnummer<br>Katalognummer<br>Número de catálogo<br>Katalog Numarası<br>Номер по каталогу |

|                                                                                      |                                                                                                                                                                                                                                                                                                                                                                                                                                                                           |                                                                                            |
|--------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------|
| LOT                                                                                  | Lot Number<br>Numéro de lot<br>Chargenbezeichnung<br>Número de lote<br>Numero di lotto                                                                                                                                                                                                                                                                                                                                                                                    | Partijnummer<br>Partinummer<br>Número de lote<br>Lot Numarası<br>Номер партии              |
|     | Use By<br>Utiliser avant<br>Verwendbar bis<br>Fecha de caducidad<br>Data di scadenza                                                                                                                                                                                                                                                                                                                                                                                      | Uiterste gebruiksdatum<br>Brukes før<br>Validade<br>Son Kullanma Tarihi<br>Использовать до |
|    | Manufacturer<br>Fabricant<br>Hersteller<br>Fabricante<br>Produttore                                                                                                                                                                                                                                                                                                                                                                                                       | Fabrikant<br>Produsent<br>Fabricante<br>Üretici Firma<br>Производитель                     |
|    | Caution<br>Attention<br>Vorsicht<br>Cuidado<br>Attenzione                                                                                                                                                                                                                                                                                                                                                                                                                 | Let op<br>Obs!<br>Cuidado<br>Dikkat<br>Внимание                                            |
|    | Do not resterilize<br>Ne pas restériliser<br>Nicht resterilisieren<br>No lo vuelva a esterilizar<br>Non ristertilizzare<br>Niet hersterilisieren<br>Må ikke steriliseres igjen<br>Não reesterilizar<br>Tekrar sterilize etmeyin<br>Не стерилизовать повторно                                                                                                                                                                                                              |                                                                                            |
|    | Do Not Reuse<br>Ne pas réutiliser<br>Nicht wiederverwenden<br>No reutilizar<br>Non riutilizzare<br>Niet opnieuw gebruiken<br>Ikke til gjenbruk<br>Não reutilizar<br>Bir Defadan Fazla Kullanmayın<br>Не использовать повторно                                                                                                                                                                                                                                             |                                                                                            |
|  | Do not use if the packaging is damaged<br>Ne pas utiliser si le conditionnement est endommagé<br>Bei beschädigter Verpackung nicht verwenden<br>No debe utilizarse si el envase está dañado<br>Non utilizzare se la confezione è danneggiata<br>Niet gebruiken als verpakking beschadigd is<br>Må ikke brukes hvis emballasjen er skadet<br>Não usar se a embalagem estiver danificada<br>Ambalaj hasarlıysa kullanmayın<br>Не использовать в случае повреждения упаковки |                                                                                            |
|  | Consult Instructions for Use<br>Consulter le mode d'emploi !<br>Gebrauchsanleitung konsultieren!<br>Consulte las Instrucciones de uso<br>Consultare le Istruzioni per l'uso<br>Raadpleeg de gebruiksaanwijzing!<br>Se bruksanvisningen!<br>Consultar as Instruções de Utilização!<br>Kullanma Talimatına Danışın<br>Следуйте инструкции по эксплуатации                                                                                                                   |                                                                                            |
|  | Not made with natural rubber latex<br>Ne contient pas de latex<br>Nicht aus Naturkautschuklatex gefertigt<br>No contiene látex de goma natural<br>Non contiene lattice di gomma naturale<br>Niet vervaardigd met natuurlijke rubberlatex<br>Ikke produsert med lateks av naturgummi<br>Não contém látex de borracha natural<br>Doğal kauçuk lateks ile yapılmamıştır<br>Не содержит натуральный каучуковый латекс                                                         |                                                                                            |
| STERILE EO                                                                           | Sterilized using EthyleneOxide<br>Stérilisé à l'oxyde d'éthylène<br>Mit Ethylenoxid sterilisiert<br>Esterilizado mediante óxido de etileno<br>Sterilizzato con ossido di etilene<br>Gesteriliseerd met ethyleenoxide<br>Steriliseret med etylenoksid<br>Esterilizado por óxido de etileno<br>Etilen Oksit Kullanılarak Sterilize Edilmiştir<br>Стерилизовано этиленоксидом                                                                                                |                                                                                            |

|                                      |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |
|--------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <div>EC</div> <div>REP</div>         | Authorized Representative in the European Community<br>Représentant agréé dans l'Union européenne<br>Autorisierte EU-Vertretung<br>Representante autorizado en la comunidad europea<br>Rappresentante autorizzato per l'Unione Europea<br>Erkend vertegenwoordiger in de Europese Gemeenschap<br>Autorisert representant i EU<br>Representante Autorizado na Comunidade Europeia<br>Avrupa Topluluğu İçindeki Yetkili Temsilci<br>Уполномоченный представитель в Европейском сообществе |
| <div>MD</div>                        | Medical device<br>Dispositif médical<br>Medizinprodukt<br>Dispositivo médico<br>Dispositivo medico<br>Medisch hulpmiddel<br>Medisinsk utstyr<br>Dispositivo médico<br>Tibbi cihaz<br>Медицинский Прибор                                                                                                                                                                                                                                                                                 |
| <div></div>                          | Single Sterile Barrier System<br>Système de barrière stérile unique<br>Einfaches Sterilbarriersystem<br>Sistema de barrera estéril individual<br>Sistema di barriera sterile singola<br>Systeem met enkele steriele barrière<br>Enkelt sterilt barriersystem<br>Sistema de Barreira Estéril Única<br>Tek Steril Bariyer Sistemi<br>Стерильная барьерная система с одним наконечником                                                                                                    |
| <div><div>A</div>→<div>文</div></div> | <div>Translation</div> <div>Traduction</div> <div>Übersetzung</div> <div>Traducción</div> <div>Traduzione</div> <div>Vertaling</div> <div>Oversettelse</div> <div>Tradução</div> <div>Çeviri</div> <div>Перевод</div>                                                                                                                                                                                                                                                                   |
| <div><div></div><div>MX</div></div>  | Country of Manufacture<br>Pays de fabrication<br>Herstellungsländ<br>Pais de fabricación<br>Paese di produzione<br>Land waar de goederen vervaardigd zijn<br>Produktionsland<br>Pais de fabrico<br>Üretim Ülkesi<br>Страна изготовления                                                                                                                                                                                                                                                 |
| <div>UDI</div>                       | Unique Device Identification<br>Identifiant unique du dispositif<br>Einmalige Produktkennung<br>Identificador exclusivo del dispositivo<br>Identificazione univoca del dispositivo<br>Unieke code voor de identificatie van hulpmiddelen<br>Unik utstyrsidentifikasjon<br>Identificação Única do Dispositivo<br>Benzersiz Cihaz Kimliği<br>Уникальная идентификация устройства                                                                                                          |

## Applicator Tip

 Before using, read the following information:

\*\*\*\*IMPORTANT NOTE\*\*\*\*  
This device is intended for use with an Applicator Assembly which is **not included**. Applicator Assemblies must be ordered **separately**. Make sure you have both parts before proceeding.

### DESCRIPTION/INDICATIONS FOR USE:

Sterile, single use instrument for the application of two liquids.

### CONTRAINDICATIONS:

None known.

### WARNINGS \ PRECAUTIONS:

Device is intended for use by Medical Professionals.

If sterile packaging is damaged or unintentionally opened prior to use, do not use the device and dispose per the Disposal instructions below.

Device is intended for single use only. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness or death of a patient.

There is a low probability that a delay in treatment, increased procedural time, mild tissue damage or local immunogenic response could occur if the device fails to function as intended.

### INSTRUCTIONS FOR USE:

1. REMOVE Applicator Tip from its sterile package using sterile technique.
2. REMOVE the Applicator Assembly from its sterile package.
3. Twist Applicator Assembly syringes firmly onto luer-lock hubs of Tip. (A). (See Applicator Assembly Instructions.)
4. COMPLETE ASSEMBLY according to Applicator Assembly Instructions.
5. Applicator Tip is ready for use.

**NOTE: Tips (metal Tips only) may be angled to provide improved access. (B)**

### DISPOSAL:

Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

### STORAGE CONDITIONS:

Sterilized products should be stored in a dry, clean environment, protected from direct sunlight, dust, pests, and extreme temperature and humidity.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member state in which the user/patient is established.

United States only: 1-800-624-5662

Other Countries: Contact your local representative and/or distributor.

 INTERNATIONAL LANGUAGE SERVICES  
601 Carlson Pkwy #1050  
Minnetonka, MN 55305  
800-225-8964

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