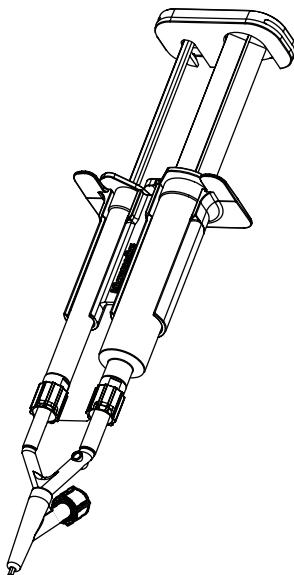
FibriJet® GAS ASSISTED APPLICATOR KIT

Instructions for Use
Mode d'emploi
Gebrauchsanweisung
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Instruções de Utilização
Инструкция по эксплуатации

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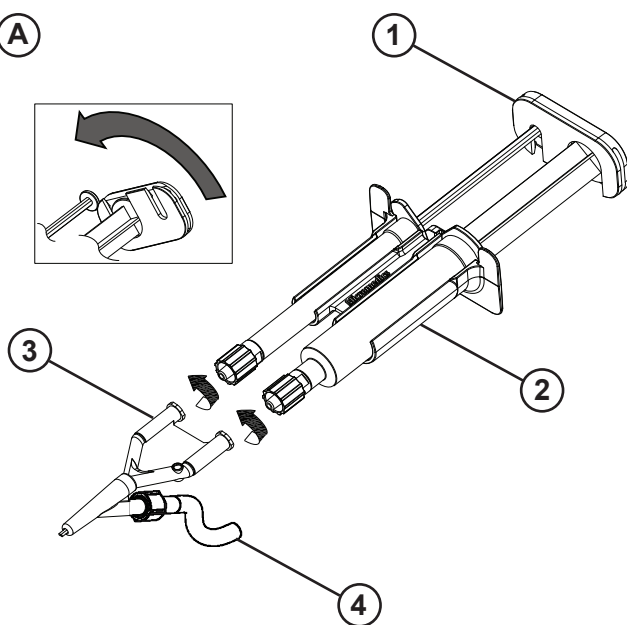
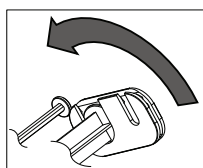
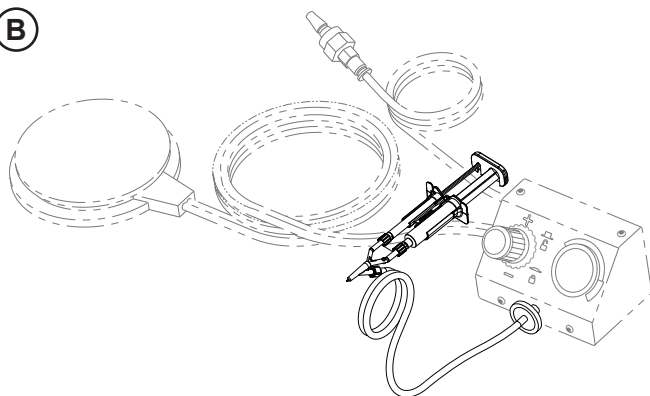
FibriJet®

Gas Assisted Applicator Kit

Before using,
read the following information:

**** IMPORTANT NOTE ****

This device must be connected to a Regulator (such as SA-6030) and is only to be used with Nitrogen, CO₂, or hospital air. Regulator is not included. It must be ordered separately. Make sure you have all parts before proceeding.

DESCRIPTION:

Each sterile, single-use FibriJet® Gas Assisted Applicator Kit contains an Applicator Assembly (1-2), Gas Assisted Spray Tip (3) and Tubing with In-Line Filter (4).

INDICATIONS FOR USE:

The FibriJet® Gas Assisted Applicator/Kit is intended for use in applying two fluids to a treatment site.

CONTRAINDICATIONS:

This device is contraindicated for intravascular use and for procedures involving insufflating a confined body cavity.

PRECAUTIONS:

Air or gas embolism has occurred with the use of spray devices employing pressure regulated gas. This event appears to be related to the use of the spray device at higher than recommended pressure and in close proximity to the tissue surface. When spraying using a pressurized spray device, be sure to use the pressure range recommended in the Instructions for Use.

To avoid possible gas embolism, do not spray directly into circulatory pathways. Any application of pressurized gas may be associated with a potential for tissue rupture or gas entrapment with compression, which may be life threatening.

Be sure to take appropriate measures to address these risks by observing these recommendations:

- Do not spray at a distance closer to the surface of tissues than 3cm at a maximum flow rate of 2 standard liters per minute.
- When using pressurized spray devices, changes in blood pressure, pulse, oxygen saturations, and end tidal CO₂ should be monitored because of the possibility of occurrence of air or gas embolism.

INSTRUCTIONS FOR SETUP:

1. REMOVE the Applicator Procedure Kit from its sterile package using sterile technique.
*Applicable only for SA-6111: For convenience, clear plastic cups and lids are provided along with colored labels to differentiate liquids used during the procedure.
2. Connect the Regulator to a gas source according to the Regulator Instructions for Use.
3. (A1 and A2) Assemble Applicator Assembly per the Applicator Assembly Instructions for Use. (A3) Twist Applicator Assembly syringes firmly onto luer lock hubs of the Applicator Tip.
4. (A4) Connect Applicator to Regulator using supplied filter tubing.

INSTRUCTIONS FOR USE:

1. Start gas flow per the Regulator Instructions for Use. Adjust pressure to 5-20 psi (0.3-1.4 bar). Dispense syringe contents by depressing plunger link using VERY SLOW STEADY PRESSURE.

To obtain even coverage, keep constant pressure on syringes and spray with sweeping motion.

To stop spray delivery, release pressure on syringe plungers while maintaining gas flow for 3-5 seconds after applying liquids to clear Applicator Tip.

2. Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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