
BIOMET[®]

BIOLOGICS

AEROSOL APPLICATOR KIT WITH TIPS

Instructions for Use
Mode d'emploi
Gebrauchsanweisung
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Instruções de Utilização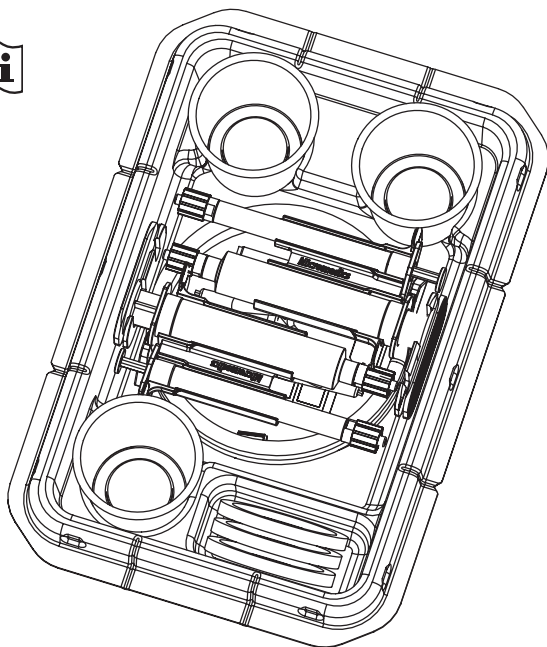

DIST

Biomet Biologics
56 East Bell Drive
P.O. Box 587
Warsaw IN 46581, USA

Manufactured in Mexico for:

Micromedics, Inc.
1270 Eagan Industrial Road
St. Paul MN 55121-1385, USA
651-452-1977



CE 0297

EC REP

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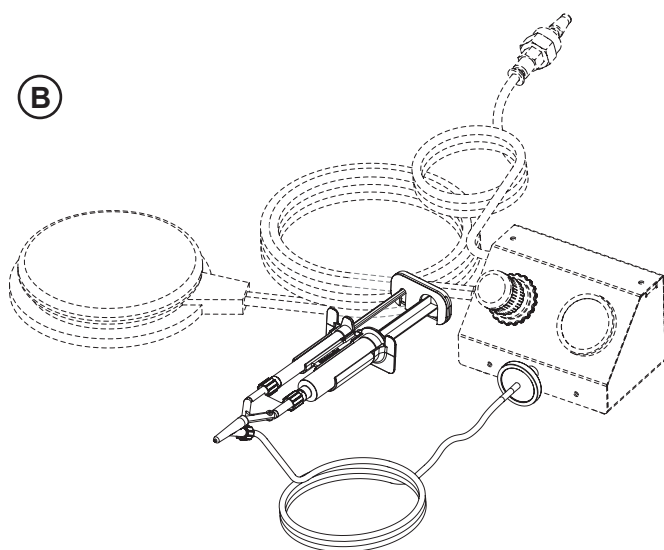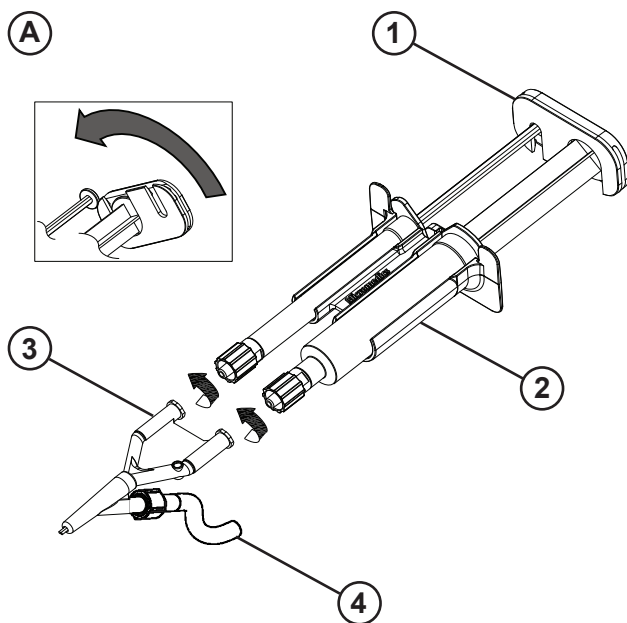AEROSOL APPLICATOR KIT WITH TIPS

Before using,
read the following information:

**** IMPORTANT NOTE ****

This device must be connected to a Regulator (such as SA-6030 shown in Figure B) and is only to be used with Nitrogen, CO₂, or hospital air. Regulator must be ordered separately. Make sure you have both parts before proceeding.

DESCRIPTION / INDICATIONS FOR USE:

The Applicator is intended for use in applying two fluids to a treatment site.

CONTRAINDICATIONS:

This device is contraindicated for intravascular use and for procedures involving insufflating a confined body cavity.

PRECAUTIONS:

Air or gas embolism has occurred with the use of spray devices employing pressure regulated gas. This event appears to be related to the use of the spray device at higher than recommended pressures and in close proximity to the tissue surface. When spraying using a pressurized spray device, be sure to use the pressure range recommended in the Instructions for Use.

To avoid possible gas embolism, do not spray directly into circulatory pathways. Any application of pressurized gas may be associated with a potential for tissue rupture or gas entrapment with compression, which may be life threatening.

Be sure to take appropriate measures to address these risks by observing these recommendations:

- Do not spray at a distance closer to the surface of tissues than 3 cm at a maximum flow rate of 2 standard liters per minute.
- When using pressurized spray devices, changes in blood pressure, pulse, oxygen saturations, and end tidal CO₂ should be monitored because of the possibility of occurrence of air or gas embolism.

INSTRUCTIONS FOR USE:

1. REMOVE the Applicator Procedure Kit from its sterile package using sterile technique.
*For convenience, clear plastic cups and lids are provided along with colored labels to differentiate liquids used during the procedure.
2. FILL the two Applicator syringes with desired volume of liquids [1ml 0-2± .07, -2.1-0± .05; 12ml 0-5.9± .18, 6-12± .48]
3. (Figure A) ASSEMBLE the syringes into holder (2) and turn the plunger clip (1) counterclockwise to secure it to the second syringe as shown. Ensure plungers are fastened in the plunger clip.
4. ASSEMBLE the syringes to the Gas Assist Spray Tip (3) by firmly twisting the syringes clockwise onto luer-lock hubs of the tip.
5. Connect Regulator to a gas source according to the Regulator Instructions for Use.
6. Connect Applicator to Regulator using supplied filter tubing (4).
7. Adjust pressure to Applicator between 5-20 psi (0.3-1.4 bar) on the Regulator.
8. (Figure B) Applicator is ready for use.
9. To use Applicator, start gas flow by depressing foot switch. Dispense syringe contents slowly into sterile gas stream by depressing plunger clip (1). To obtain even coverage, keep constant pressure on the syringes and spray with sweeping motion.
10. Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination

of a reprocessed device may lead to injury, illness, or death of a patient.

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