
Instructions for Use  
Mode d'emploi  
Gebrauchsanweisung  
Modo de empleo  
Istruzioni per l'uso  
Gebruiksaanwijzing  
Bruksanvisning

Manufactured in Mexico for: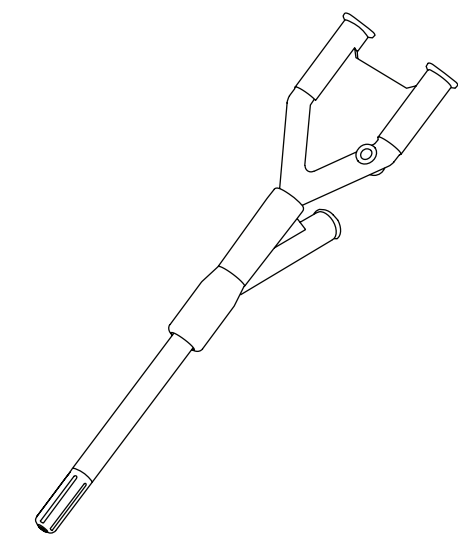

Micromedics, Inc.  
1270 Eagan Industrial Road  
St. Paul MN 55121-1385, USA  
651-452-1977  
800-624-5662  
info@nordsonmedical.com  
www.nordsonmedical.com





CE 0297

EC REP

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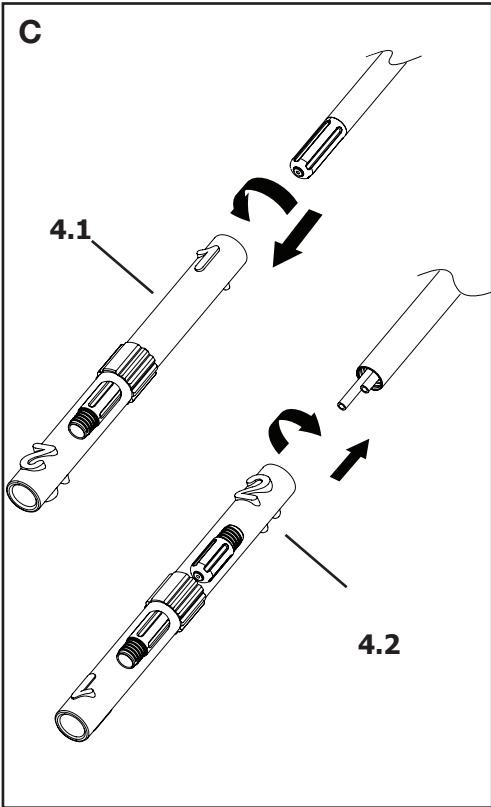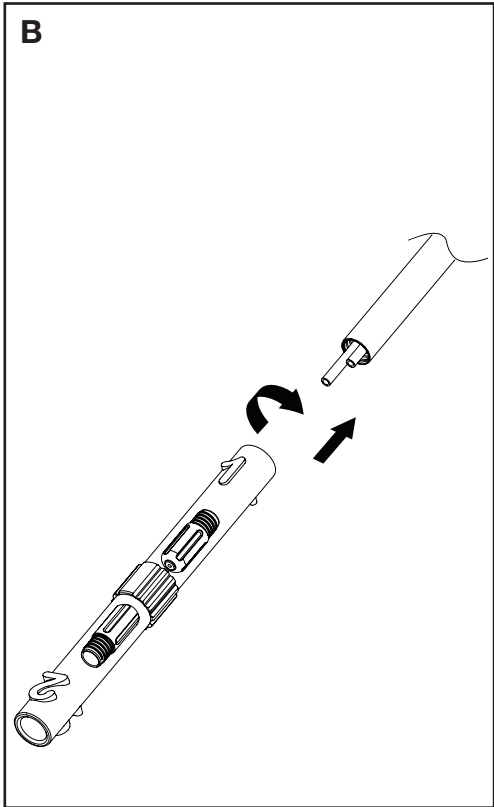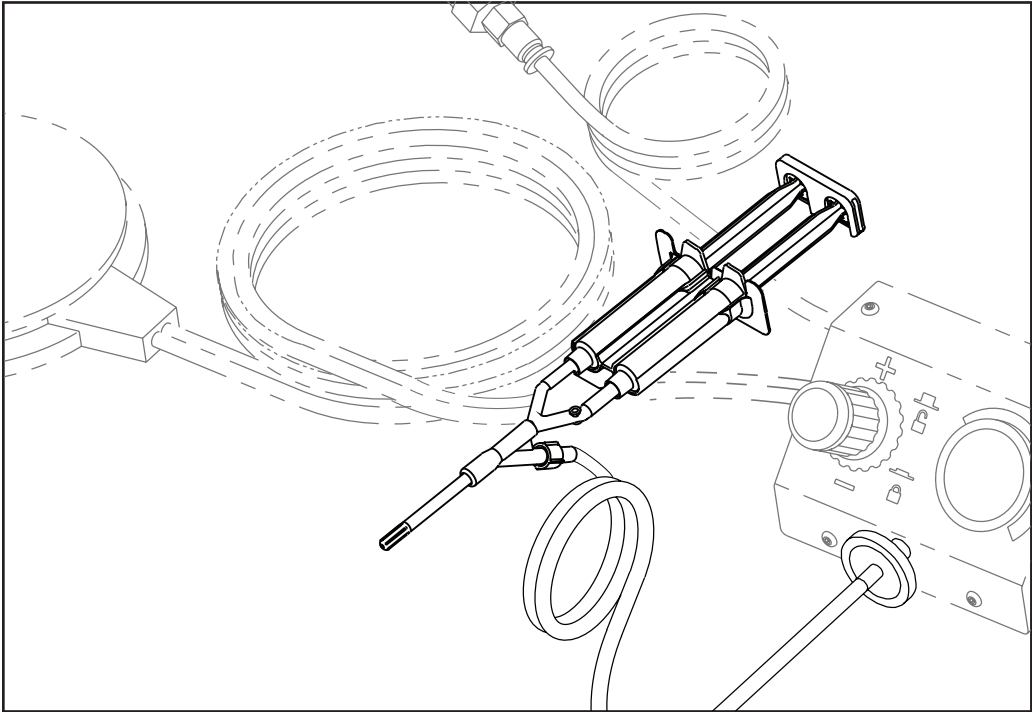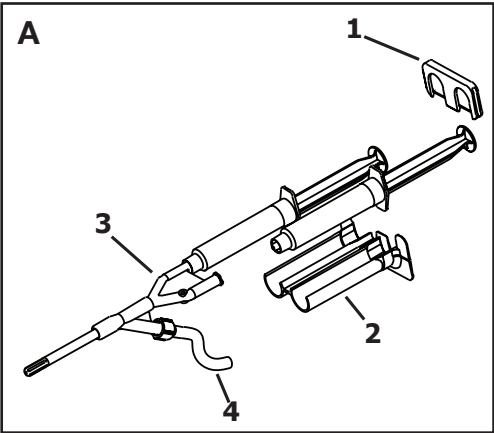EN

### Spray Set, Topical Applicator Kit With Mixing Tip

Before using, read the following information:

\*\*\*\***IMPORTANT NOTE**\*\*\*\*

***This device must be connected to a Regulator (such as SA-6030) and is only to be used with Nitrogen, CO2, or hospital air. Regulator and Applicator Assembly are not included. They must be ordered separately. Make sure you have both parts before proceeding.***

**DESCRIPTION/ INDICATIONS FOR USE:**

The FibriJet® Gas Assisted Applicator is intended for use in applying two fluids to a treatment site.

**CONTRAINDICATIONS:**

This device is contraindicated for intravascular use and for procedures involving insufflating a confined body cavity.

**PRECAUTIONS:**

Air or gas embolism has occurred with the use of spray devices employing pressure regulated gas. This event appears to be related to the use of the spray device at higher than recommended pressure and in close proximity to the tissue surface. When spraying using a pressured spray device, be sure to use the pressure range recommended in the Instructions for Use.

To avoid possible gas embolism, do not spray directly into circulatory pathways. Any application of pressurized gas may be associated with a potential for tissue rupture or gas entrapment with compression, which may be life threatening.

Be sure to take appropriate measures to address these risks by observing these recommendations:

- Do not spray at a distance closer to the surface of tissues than 3cm at a maximum flow rate of 2 standard liters per minute.
- When using pressurized spray devices, changes in blood pressure, pulse, oxygen saturations, and end tidal CO2 should be monitored because of the possibility of occurrence of air or gas embolism.

**INSTRUCTIONS FOR SETUP:**

- Connect the Regulator to a gas source according to the Regulator Instructions for Use.
- (A1 and A2) Assemble Applicator Assembly per the Applicator Assembly Instructions for Use. (A3) Twist Applicator Assembly syringes firmly onto luer lock hubs of the Applicator Tip. (B) Thread the sterile Replacement Spray Tip using the Tip Alignment Tool.
- (A4) Connect Applicator to Regulator using supplied filter tubing.

**INSTRUCTIONS FOR USE:**

- Start gas flow per the Regulator Instructions for Use. Dispense syringe contents by depressing plunger link using VERY SLOW STEADY PRESSURE. To obtain even coverage, keep constant pressure on syringes and spray with sweeping motion. To stop spray delivery, release pressure on syringe plungers while maintaining gas flow for 3-5 seconds after applying liquids to clear Applicator Tip.
  - (C) If Tip becomes occluded during use, as indicated by no spray or no movement of ball in gas flow gage; use Tip Alignment Tool to remove plugged Tip. Using sterile cloth, wipe any clotted material from exposed tube ends. Use opposite chamber of Tip Alignment Tool to install new tip and ensure it is securely fastened.
- Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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