
APPLICATOR ASSEMBLY PROCEDURE KIT WITH SPRAY TIPS

Instructions for Use
Mode d'emploi
Gebrauchsanweisung
Modo de empleo
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Instruções de Utilização

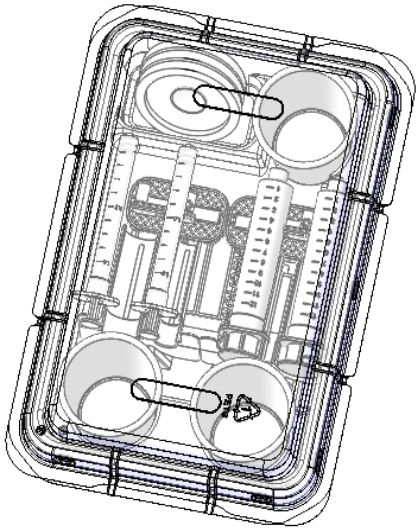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

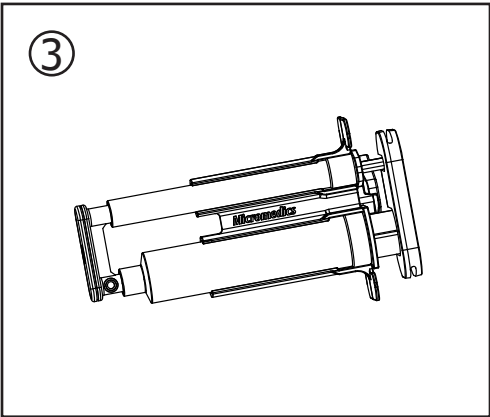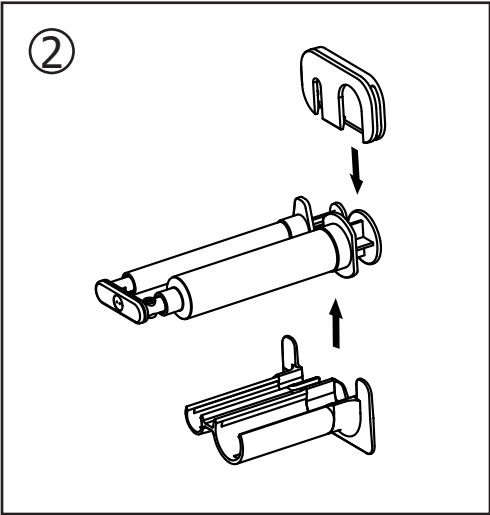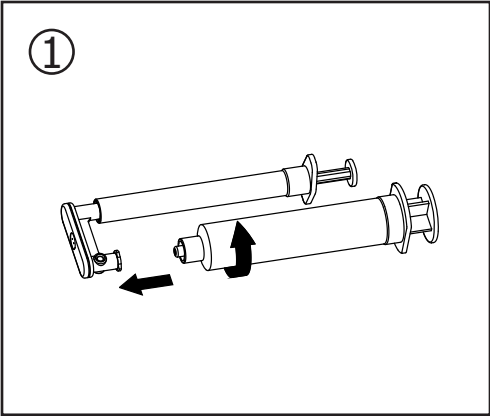
Applicator Assembly Procedure Kit with Spray Tips

⚠ Before using, read the following information:

DESCRIPTION / INDICATIONS FOR USE:

Sterile, single-use instrument for application of two liquids.

CONTRAINDICATIONS:

None known.

PRECAUTIONS:

Intended for single use only, do not resterilize.

INSTRUCTIONS FOR USE:

1. REMOVE Applicator Assembly from sterile package. PLACE tray in sterile field. Cups may be used to hold liquids to be applied.
*For convenience, clear plastic cups and lids are provided along with colored labels to differentiate liquids used during the procedure.
2. FILL Applicator syringes with desired volumes of liquids.
3. (1) ASSEMBLE syringes to FibriJet Applicator Tip by twisting syringes onto luer-lock hubs.
4. (2) ASSEMBLE holder and plunger link to syringes as shown.
5. (3) Applicator is ready for use.
6. Applicator may be refilled for subsequent uses **in same procedure only.**

To Refill Applicator follow procedure below:

- a. Remove plunger link.
 - b. Remove syringes from holder.
 - c. Remove Applicator Tip from syringes by twisting both syringes counter clockwise.
 - d. Refill syringes.
 - e. Reinstall Applicator Tip as described above.
- IMPORTANT: To avoid clogging applicator, always connect larger syringe to port with round indicator.**

7. Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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© Exactech®