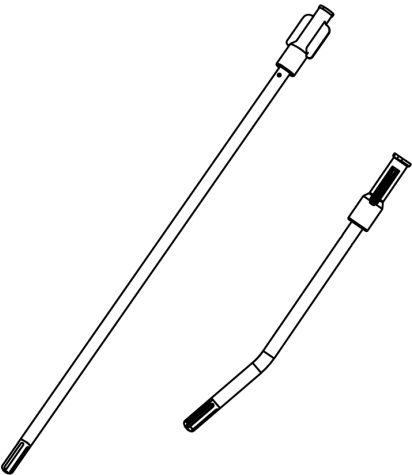
**Instructions for Use**  
**Mode d'emploi**  
**Gebrauchsanweisung**  
**Instrucciones de uso**  
**Istruzioni per l'uso**  
**Gebruiksaanwijzing**  
**Bruksanvisning**  
**Instruções de Utilização**

**MANUFACTURED FOR:**

  
Micromedics, Inc.  
1270 Eagan Industrial Road  
St. Paul MN 55121-1385, USA  
651-452-1977





**CE 0297**

**EC REP**

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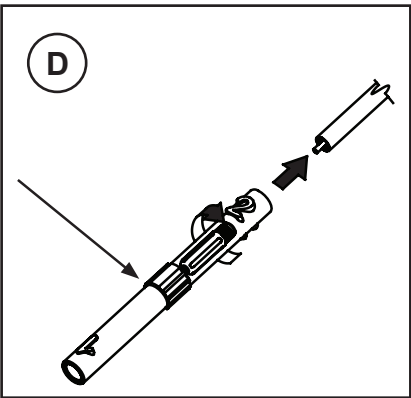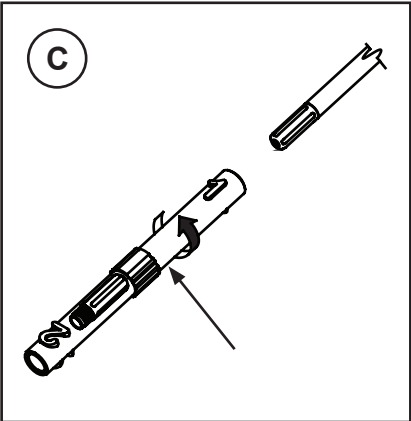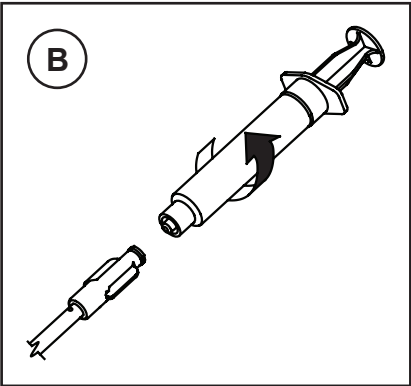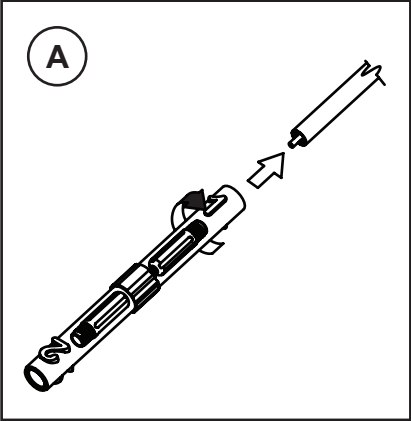**EN**

**Single Cannula Extended Applicator**

 Before using, read the following information:

**\*\*\*\*IMPORTANT NOTE\*\*\*\***

This device is for use with a syringe or compatible applicator with a standard luer lock (Applicator Assembly) which is not included. This must be ordered separately. Make sure you have both pairs before proceeding

**INTENDED USE:**

The Single Cannula Extended Applicator with Spray Tip is intended for medical purposes to irrigate or instill fluids to a wound or body cavity.

**CONTRAINDICATIONS:**

None

**WARNINGS \ PRECAUTIONS:**

If sterile packaging is damaged or unintentionally opened prior to use, do not use the device and dispose per the Disposal instructions below

Device is intended for single use only. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness or death of a patient.

There is a low probability that a delay in treatment, increased procedural time, mild tissue damage or local immunogenic response could occur if the device fails to function as intended.

- INSTRUCTIONS FOR SETUP:**
- Open sterile package and remove contents using sterile technique.
  - Use Tip Alignment Tool to thread sterile Replaceable Tip onto Applicator until seated against end of Applicator Shaft (A).  
Retain Tip Alignment Tool with second Replaceable Tip for use if first tip becomes occluded.
  - Assemble filled syringe to Applicator (B).
  - Applicator is ready for use.

- INSTRUCTIONS FOR USE:**
- (NOTE: IF LEVER FORCE IS APPLIED TO THE APPLICATOR GRASP THE UPPER PORTION OF THE STAINLESS STEEL SHAFT. DO NOT USE THE SYRINGE AS A HOLD POINT.)
- Dispense liquid into Applicator Tip by depressing syringe plunger using STEADY PRESSURE. To stop flow, release pressure on syringe plunger.
  - If Tip becomes occluded, use Tip Alignment Tool to remove Tip by unscrewing counter-clockwise (C). Use a sterile cloth to wipe any clotted material from exposed tube end. Use opposite chamber of Tip Alignment Tool to install new tip and tighten securely with finger pressure (D).

**DISPOSAL:**

Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

**STORAGE CONDITIONS:**

Sterilized products should be stored in a dry, clean environment, protected from direct sunlight, dust, pests, and extreme temperature and humidity.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member state in which the user/patient is established.

United States only: 1-800-624-5662

Other Countries: Contact your local representative and/or distributor.

 INTERNATIONAL LANGUAGE SERVICES  
601 CARLSON PKWY #1050  
MINNETONKA, MN 55305  
800-225-8964

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