
FibriJet®
Dual Spray Procedure Kit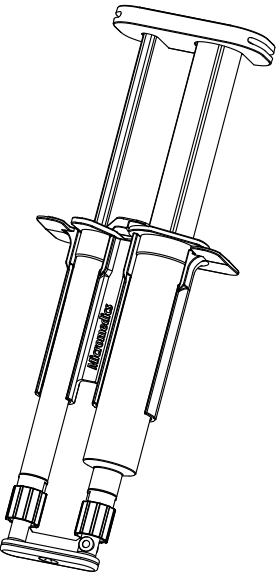





Instructions for Use ⓘ
Mode d'emploi
Gebrauchsanweisung
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Instruções de Utilização

Manufactured in Mexico for:

Micromedics, Inc.
1270 Eagan Industrial Road
St. Paul MN 55121-1385, USA
651-452-1977

CE 0297

EC REP
MDSS GmbH
Schiiffgraben 41
D-30175 Hannover, Germany

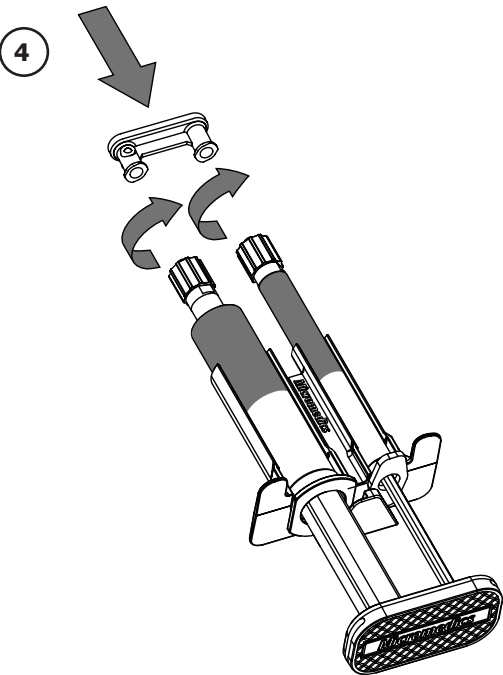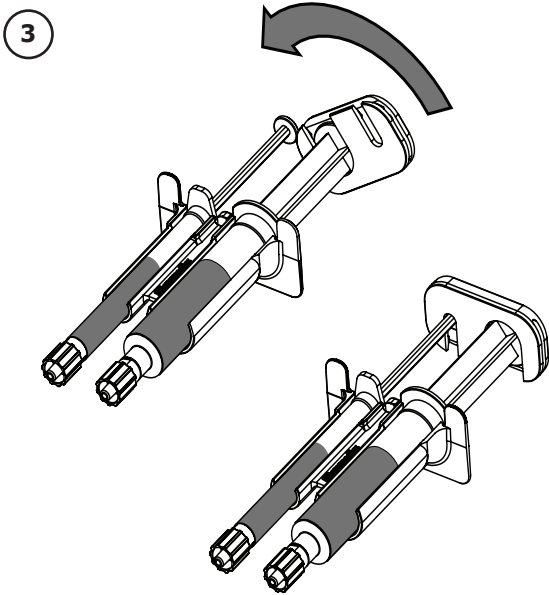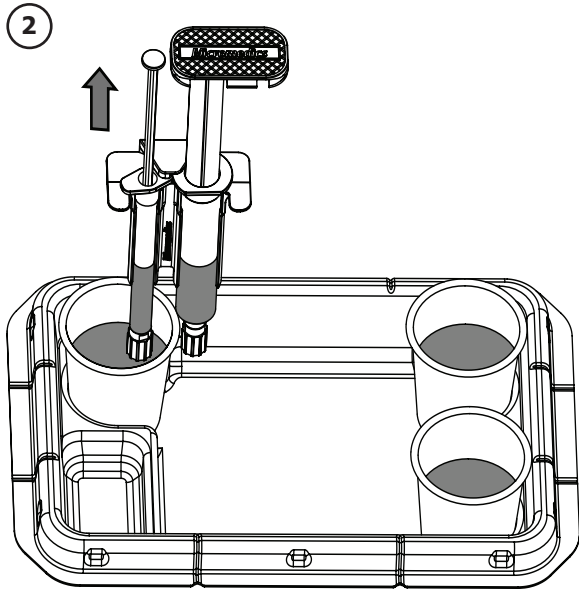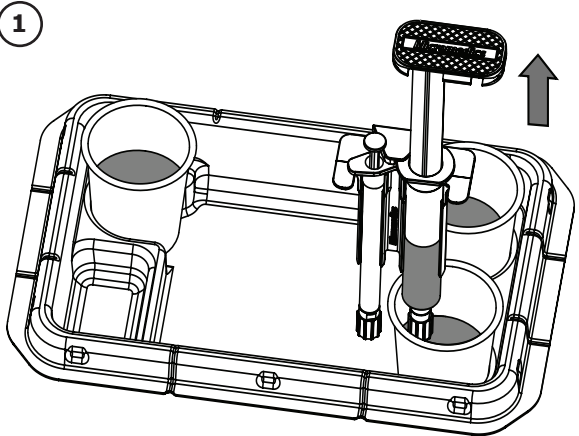Symbol Reference Key \ Légende des symboles utilisés \ Erklärung der Symbole \ Clave de referencia a los símbolos \ Legenda dei simboli \ Verklaring van symbolen \ Symbol referansen nøkkelen \ Legenda de referência dos símbolos	
DUAL SPRAY PROCEDURE KIT	
Dual Spray procedure Kit Kit de procédure du pulvérisateur double Doppelsprühapplikator-Verfahrenskit Kit para procedimiento de pulverización doble Kit procedurale a doppio spruzzo Dubbel sproeiende procedurekit Dobbelt spraysett Conjunto de procedimento de nebulização dupla	
Manufactured in Mexico for:	
Manufactured in Mexico for: Fabriqué au Mexique pour : Hergestellt in Mexiko für: Hecho en México para: Prodotto in Messico per: Vervaardigd in Mexico voor: Produsert i Mexico for: Fabricado no México para:	
	Not made with natural rubber latex Ne contient pas de latex Nicht aus Naturkautschuklatex gefertigt No está fabricado con látex de goma natural Non contiene lattice di gomma naturale Niet vervaardigd met natuurlijke rubberlatex Ikke produsert med lateks av naturgummi Não contém látex de borracha natural
REF	Catalog Number Numéro de référence Katalognummer Número de catálogo Numero di catalogo Catalogusnummer Katalognummer Número de catálogo

	Manufacturer Fabricant Hersteller Fabricante	Produttore Fabrikant Produsent Fabricante
	Caution Attention Vorsicht Cuidado	Attenzione Let op Obs! Cuidado
	Consult Instructions for Use Consulter le mode d'emploi Gebrauchsanleitung konsultieren Consulte las Instrucciones de uso Consultare le Istruzioni per l'uso Raadpleeg de gebruiksaanwijzing Se bruksanvisningen Consultar as Instruções de utilização	
LOT	Lot Number Numéro de lot Chargenbezeichnung Número de lote	Numero di lotto Partijnummer Partinummer Número de lote
	Use By Utiliser avant Verwendbar bis Fecha de caducidad	Data di scadenza Uiterste gebruiksdatum Brukes før Validade
STERILEEO	Sterilized using EthyleneOxide Stérilisé à l'oxyde d'éthylène Mit Ethylenoxid sterilisiert Esterilizado por óxido de etileno Sterilizzato con ossido di etilene Gesteriliseerd met ethyleenoxide Sterilisert med etylenoksid Esterilizado por óxido de etileno	
EC REP	Authorized Representative in the European Community Représentant agréé dans l'Union européenne Autorisierte EU-Vertretung Representante autorizado en la Comunidad Europea Rappresentante autorizzato per l'Unione Europea Erkend vertegenwoordiger in de Europese Gemeenschap Autorisert representant i EU Representante autorizado na Comunidade Europeia	
	Do not use if the packaging is damaged Ne pas utiliser si le conditionnement est endommagé Bei beschädigter Verpackung nicht verwenden No debe utilizarse si el envase está dañado Non utilizzare se la confezione è danneggiata Niet gebruiken als verpakking beschadigd is Skal ikke brukes hvis emballasjen er skadet Não utilizar se a embalagem estiver danificada	
	Do Not Reuse Ne pas réutiliser Nicht wiederverwenden No reutilizar	Non riutilizzare Niet opnieuw gebruiken Skal ikke gjenbrukes Não reutilizar
	Do not resterilize Ne pas restériliser Nicht resterilisieren No lo vuelva a esterilizar	Non risterilizzare Niet hersteriliseren Skal ikke steriliseres på nytt Não reesterilizar

EN

FibriJet®
Dual Spray Procedure Kit

Before using, read the following information:

DESCRIPTION/ INDICATIONS FOR USE:
Sterile, single-use instrument for application of two liquids.

CONTRAINDICATIONS:
None known

PRECAUTIONS:
IMPORTANT: To avoid clogging applicator, always connect larger syringe to port with the round indicator.

INSTRUCTIONS FOR USE:

- REMOVE the Applicator Procedure Kit from its sterile package using sterile technique. *For convenience, clear plastic cups and lids are provided along with colored labels to differentiate liquids used during the procedure.
- (Figure 1 and Figure 2) FILL the two Applicator syringes with desired volume of liquids [1ml 0-2± .07, .2-1.0± .05; 12ml 0-5.9± .18, 6-12± .48].
- (Figure 3) ASSEMBLE the syringes into holder and turn the plunger clip counterclockwise to secure it to the second syringe as shown. Ensure plungers are fastened in the plunger clip.
- (Figure 4) Twist Applicator Assembly syringes firmly onto luer lock hubs of the Applicator Tip.
- Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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