
ARTHROSCOPIC DELIVERY KIT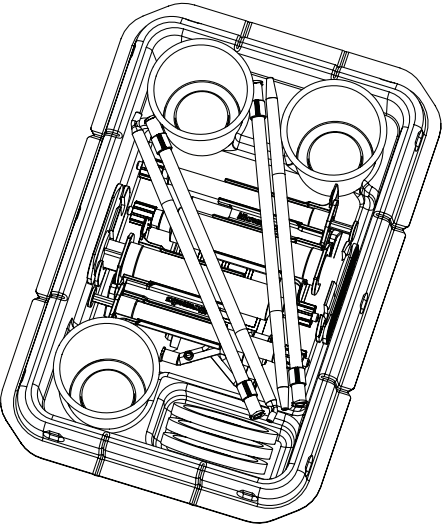

Instructions for Use

Mode d'emploi
Gebrauchsanweisung
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Instruções de Utilização

DIST

56 EAST BELL DRIVE
P.O. BOX 587
WARSAW, IN 46581 USA

Manufactured in Mexico for:

Micromedics, Inc.
1270 Eagan Industrial Road
St. Paul, MN 55121-1385 USA
651-452-1977

CE 0297






EC REP

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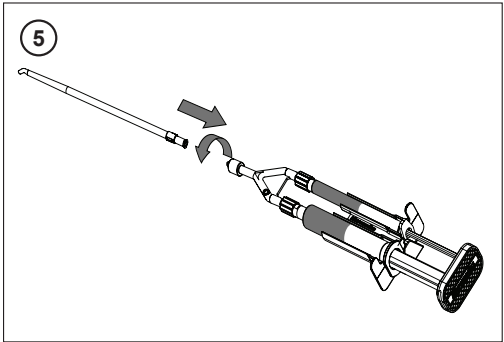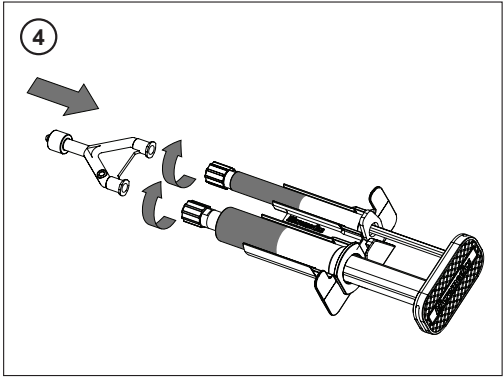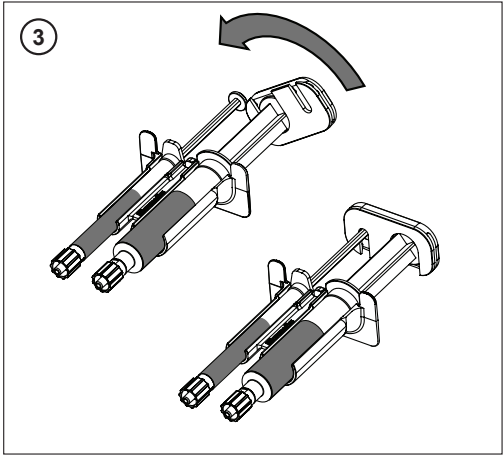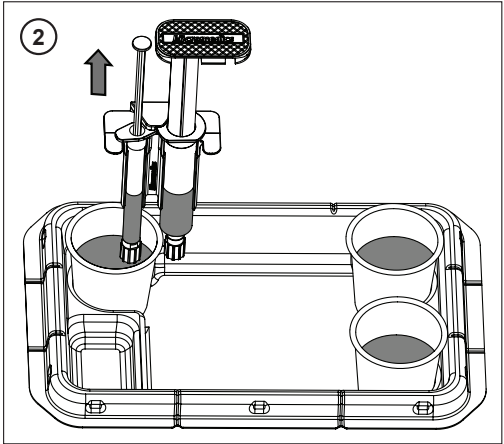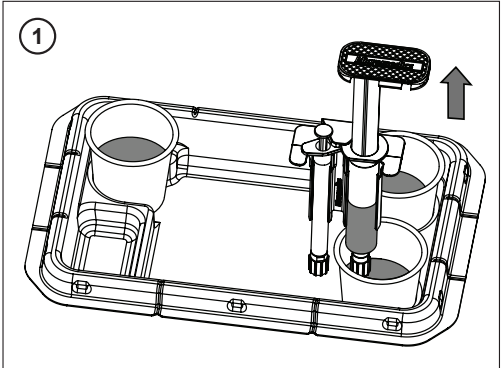ARTHROSCOPIC DELIVERY KIT

Before using, read the following information:

DESCRIPTION/INDICATIONS FOR USE

Sterile, single-use instrument for application of two liquids.

CONTRAINDICATIONS

None Known

PRECAUTIONS

IMPORTANT: To avoid clogging applicator, always connect larger syringe to port with the round indicator.

INSTRUCTIONS FOR USE

- REMOVE the Applicator Procedure Kit from its sterile package using sterile technique.
- (Figure 1 and Figure 2) FILL the two Applicator syringes with desired volume of liquids [1ml 0-2± .07, .2-1.0± .05; 12ml 0-5.9± .18, 6-12± .48]
- (Figure 3) ASSEMBLE the syringes into holder and turn the plunger clip counterclockwise to secure it to the second syringe as shown. Ensure plungers are fastened in the plunger clip.
- (Figure 4) ASSEMBLE the syringes to the Applicator Tip by firmly twisting the syringes clockwise onto luer-lock hubs of the tip.
- (Figure 5) Twist the Mixing Tip clockwise onto a standard luer lock (i.e., syringe). Ensure spray tip is secured to prevent accidental detachment.
- Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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