FibriJet®

RATIO PROCEDURE KIT

Instructions for Use

Mode d'emploi

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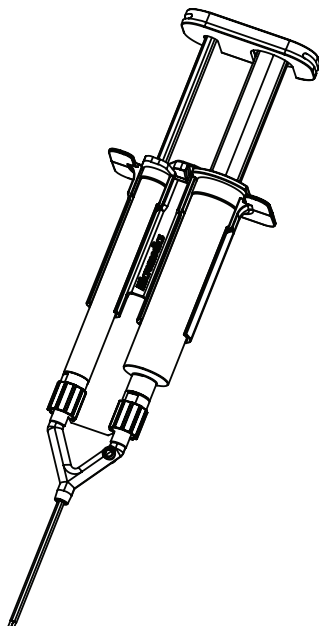
Bruksanvisning

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Manufactured in Mexico for:

Manufacturer | Micromedics, Inc.
1270 Eagan Industrial Road
St. Paul MN 55121-1385, USA
651-452-1977

CE 0297


EC REP

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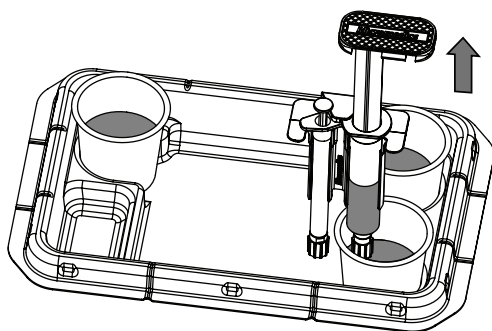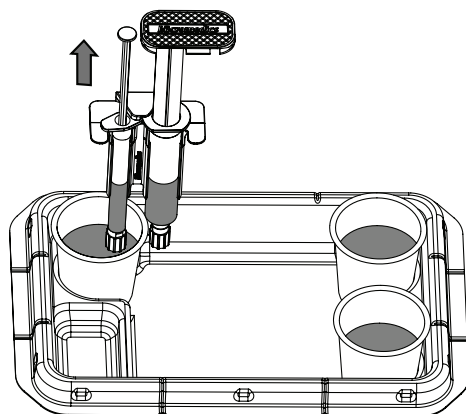
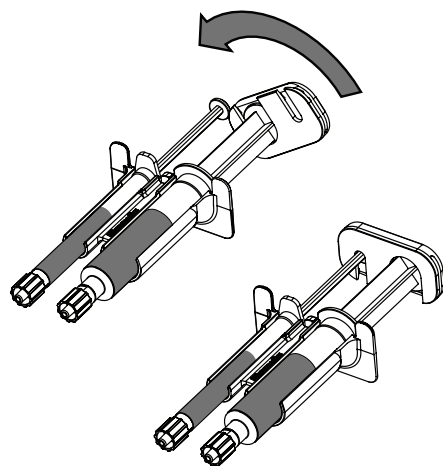
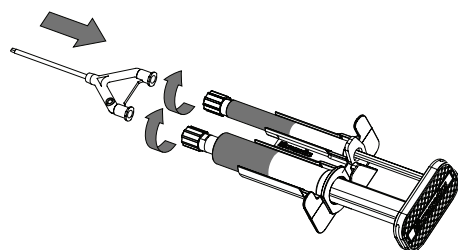
FibriJet®

Ratio Procedure Kit

Before using, read the following information:

**** IMPORTANT NOTE ****

This device is intended for use with an applicator tip which is not included. Applicator tips must be ordered separately. Make sure you have both parts before proceeding.

DESCRIPTION / INDICATIONS FOR USE:

Sterile, single-use instrument for application of two liquids.

CONTRAINDICATIONS:

None known

PRECAUTIONS:

IMPORTANT: To avoid clogging applicator, always connect larger syringe to port with the round indicator.

INSTRUCTIONS FOR USE:

1. REMOVE the Applicator Procedure Kit from its sterile package using sterile technique.
*For convenience, clear plastic cups are provided along with colored labels to differentiate liquids used during the procedure.
2. (Figure 1 and Figure 2) FILL the two Applicator syringes with desired volume of liquids [1ml 0-2± .07, 2-1.0± .05; 12ml 0-5.9± .18, 6-12± .48]
3. (Figure 3) ASSEMBLE the syringes into holder and turn the plunger clip counterclockwise to secure it to the second syringe as shown. Ensure plungers are fastened in the plunger clip.
4. (Figure 4) ASSEMBLE the syringes to the Applicator Tip by firmly twisting the syringes clockwise onto luer-lock hubs of the tip.
5. Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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