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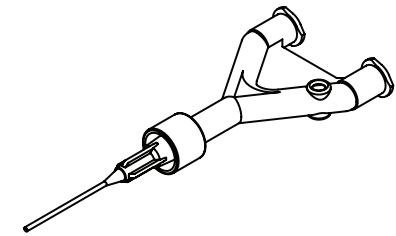
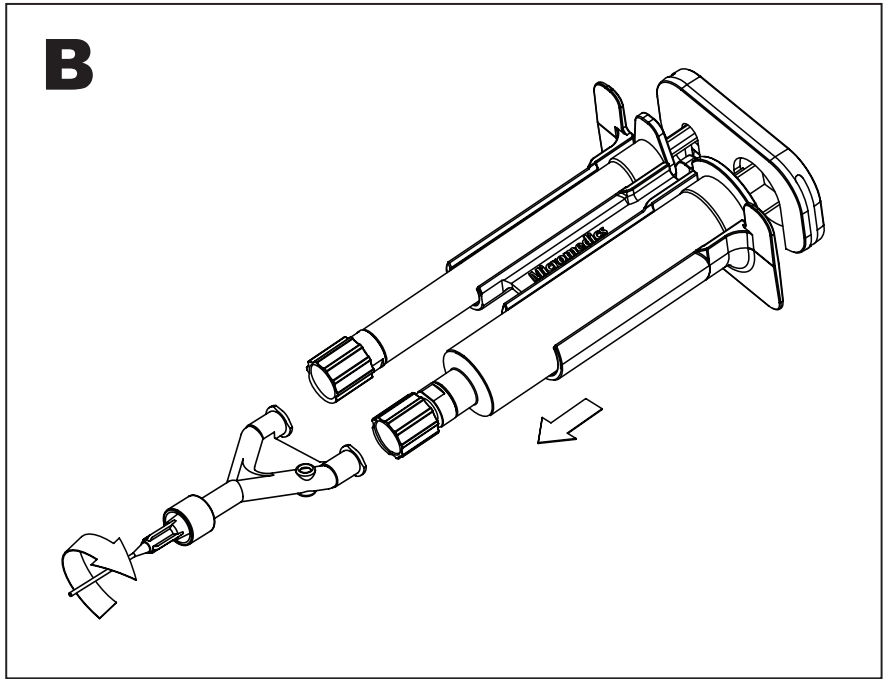
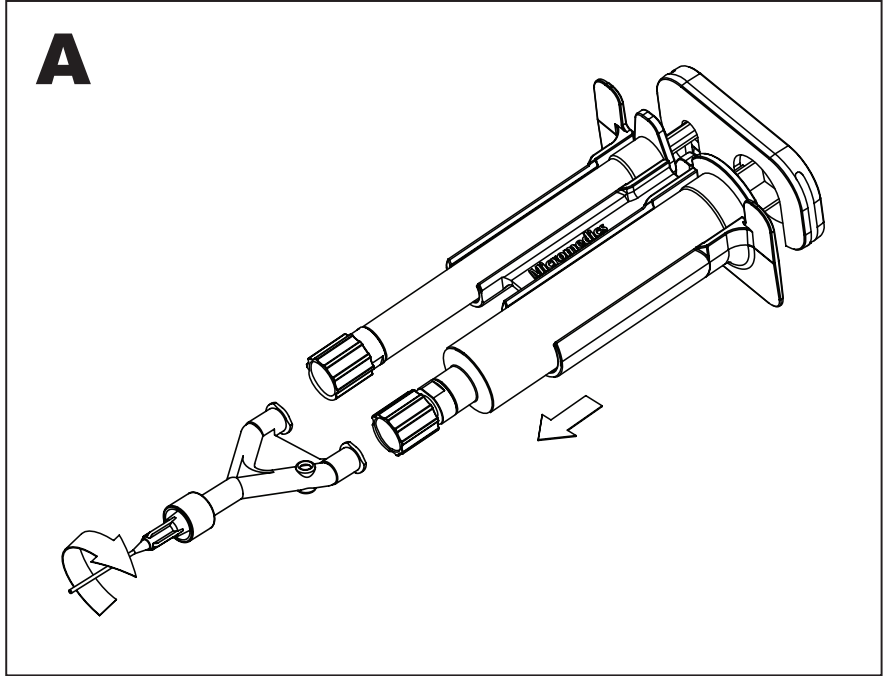
BLENDING CONNECTOR TIP WITH SINGLE CANNULA

Before using, read the following information:

****IMPORTANT NOTE****
This device is intended for use with an Applicator Assembly which is not included. Applicator Assemblies must be ordered separately. Make sure you have both parts before proceeding.

DESCRIPTION/INDICATIONS FOR USE

Sterile, single-use instrument for application of two liquids.

CONTRAINDICATIONS

None known.

PRECAUTIONS

This device is intended for single use only, do not resterilize.

INSTRUCTIONS FOR USE

- REMOVE Applicator Tip from its sterile package using sterile technique.
- REMOVE the Applicator Assembly from its sterile package.
- See Applicator Assembly Instructions (FIG. A).
- ASSEMBLE included Spray Tip or Cannula Tip to Blending Connector by twisting firmly clockwise onto luer-lock connection (FIG. B).
- COMPLETE ASSEMBLY according to Applicator Assembly Instructions.
- Applicator Tip is ready for use.
- Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

NOTE: To avoid clogging, always put syringe containing Fibrinogen or Plasma on side with round indicator.

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