
SPRAY APPLICATOR KIT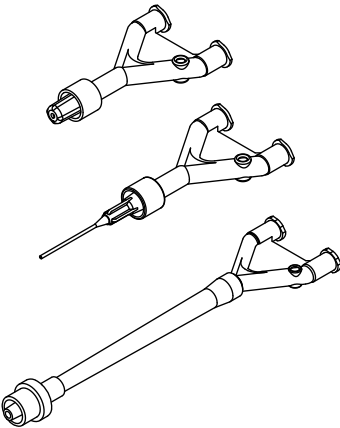

Instructions for Use
Mode d'emploi
Gebrauchsanweisung
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Instruções de Utilização

DIST

800-12 Capitola Drive
Durham, NC 27713-4410 USA

Manufactured in Mexico for:

Micromedics, Inc.
1270 Eagan Industrial Road
St. Paul MN 55121-1385, USA
651-452-1977
800-624-5662



CE 0297

EC REP

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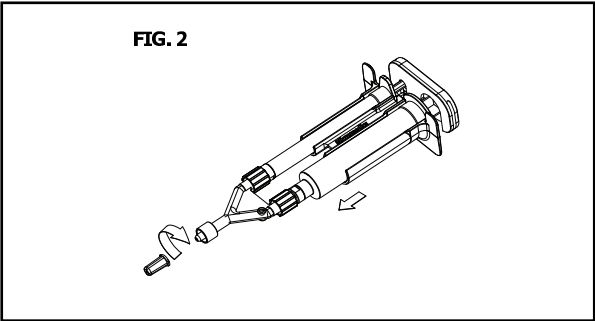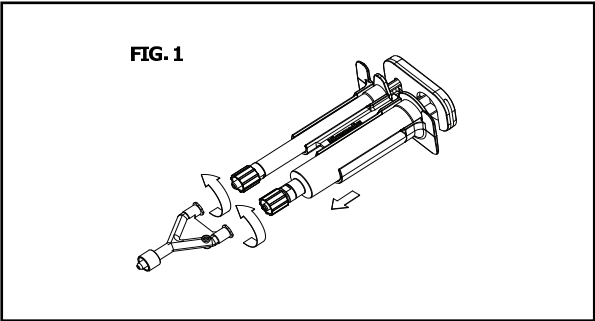FibriJet
Spray Applicator Kit

Before using, read the following information:

****IMPORTANT NOTE****
This device is intended for use with an Applicator Assembly which is not included. Applicator Assemblies must be ordered separately. Make sure you have both parts before proceeding.

DESCRIPTION / INDICATIONS FOR USE:

Sterile, single-use instrument for the application of two liquids.

CONTRAINDICATIONS:

None known.

PRECAUTIONS:

This device is intended for single use only, do not resterilize.

INSTRUCTIONS FOR USE:

- REMOVE Applicator Tip from its sterile package using sterile technique.
- REMOVE the Applicator Assembly from its sterile package.
- See Applicator Assembly Instructions (FIG. 1).
- ASSEMBLE included Spray Tip or Cannula Tip to Blending Connector by twisting firmly clockwise onto Luer-lock connection (FIG. 2).
- COMPLETE ASSEMBLY according to Applicator Assembly Instructions.
- Applicator Tip is ready for use.
NOTE: To avoid clogging, always put syringe containing Fibrinogen or Plasma on side with round indicator.
- Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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