
DuploSpray MIS Applicator

Nordson
MEDICAL

Instructions for Use
Mode d'emploi
Gebrauchsanleitung
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Brugsanvisning
Οδηγίες χρήσης
Instruções de Utilização
Bruksanvisning
Pokyny k použití
Instrukcja obsługi
Kullanım Talimatları
Указания за употреба
Uputstva za uporabo
Käyttöohjeet

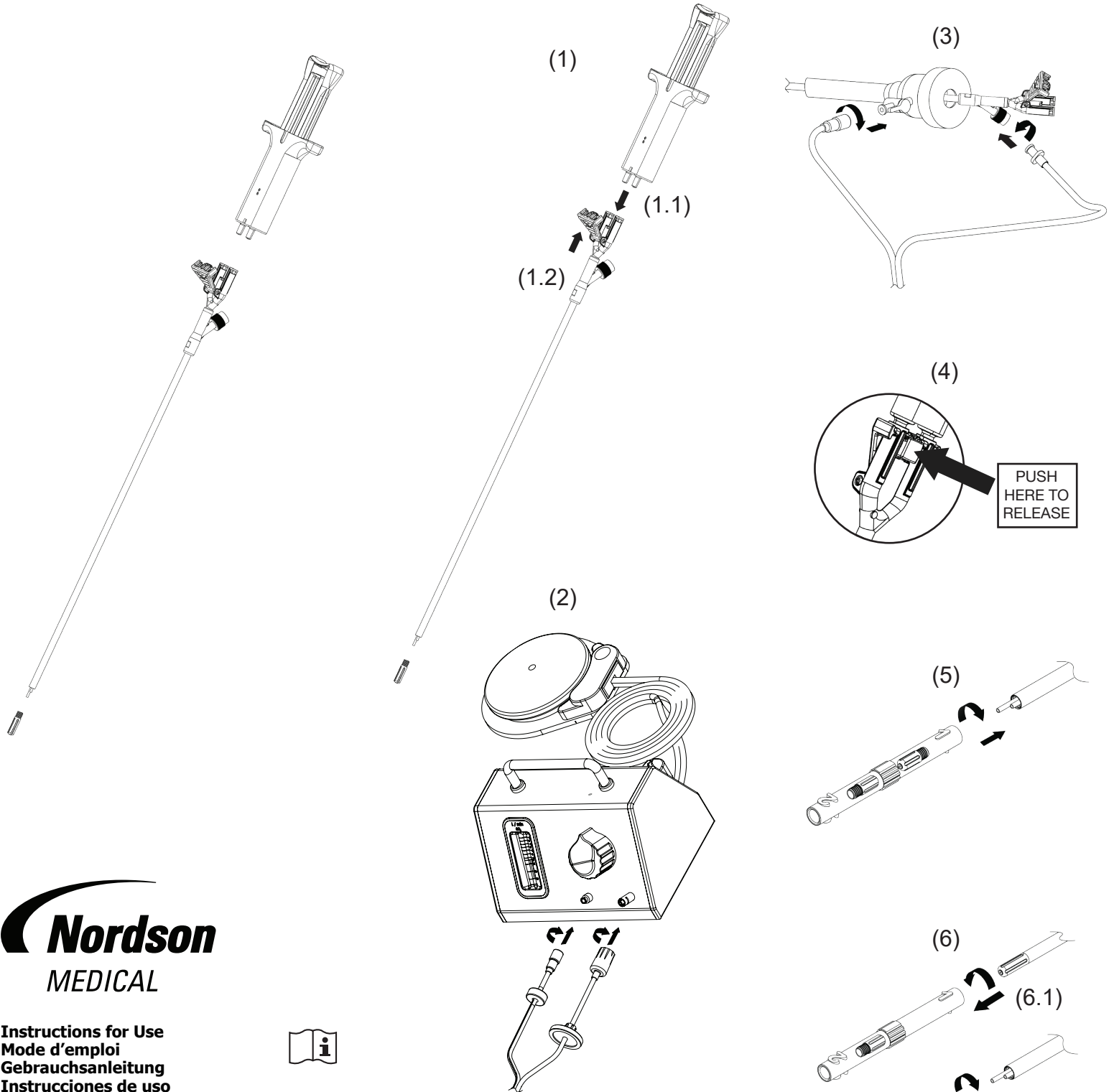Manufactured in Mexico for:

Micromedics, Inc.
1270 Eagan Industrial Road
St. Paul, MN 55121-1385 USA
651-452-1977

CE 0297

EC REP

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Baxter Healthcare Corporation
Deerfield, IL 60015 USA
1-888-229-0001

DuploSpray MIS Applicator

Before using, read the following information:

INTENDED USE:

The DuploSpray MIS Applicator is intended for the application of TISSEEL Fibrin Sealant.

CONTRAINDICATIONS:

None.

WARNINGS

Use only with approved DuploSpray MIS Regulators. Connect regulator to regulated CO2 gas source; maximum input pressure not to exceed 100 psi (7 bar). See regulator IFU for more information.

Caution must be used when applying product using pressurized gas.

- Air or gas embolism has occurred with the use of spray devices employing a pressure regulator to administer fibrin sealants. This event appears to be related to the use of the spray device at higher than recommended pressures and in close proximity to the tissue surface. When applying sealants using a spray device, be sure to use the flow rate recommended in the Instructions for Use.
- To avoid possible gas embolism, do not spray directly into circulatory pathways. Any application of pressurized gas is associated with a potential risk of air embolism, tissue rupture or gas entrapment with compression, which may be life threatening.

Be sure to take appropriate measures to address these risks by observing these recommendations:

- Do not spray at a distance closer to the surface of tissues than 2 cm (3cm is recommended) at a maximum flow rate of 2.0 liters per minute (L/min).
- When using pressurized spray devices, changes in blood pressure, pulse, oxygen saturations, and end tidal CO2 should be monitored because of the possibility of occurrence of air gas embolism.

APPLICATOR ASSEMBLY:

- Remove the Applicator from its package using sterile technique.
- Use the Tip Alignment Tool to thread a sterile Replaceable Tip onto the Applicator until seated against the end of the Applicator shaft (Figure 5). Retain the Tip Alignment Tool, which holds a second Replaceable Tip provided for use if the first tip becomes occluded.
- Connect the Tubing Set, Regulator, and Applicator as shown in Figures 2 and 3. To connect the Tubing Set, attach the clear hooded Luer connector to the male Luer gas port on the Regulator. Connect the other clear female Luer connector to the male Luer gas port on the Applicator. Attach the blue male Luer connector on the Tubing Set to the female Luer vent port on the Regulator and attach the red male Luer connector to the female Luer connector on the Trocar cannula vent valve. Check to make sure that the Trocar vent valve is in the fully open position to allow venting. (Note that a Trocar cannula with a vent valve must be used for the vent feature to work properly.)
- Push the male syringe set Luers firmly into the female Luer cones on the Applicator as shown in Figure 1.1.
- Push down on the Snap Lock to securely fasten the Applicator to the syringe Luers as shown in Figure 1.2.

INSTRUCTIONS FOR USE:

- Press the foot switch of the DuploSpray MIS Regulator to start gas flow prior to applying Tisseel.
- Check the gas flow meter on Regulator before inserting Applicator into Trocar. (Maximum flow rate = 2.0 L/min; minimum spray distance = 2cm)

Flow Rate	1.0 - 2.0 Liters per minute (L/min)		
Distance	2 cm	3 cm	5 cm
	Recommended		

If there is no spray from the Applicator or no movement of the ball in the gas flow meter on the Regulator, the Replaceable Tip is likely occluded and should be replaced. (See step 4 for tip replacement.)

- While activating foot switch, dispense Tisseel into the Applicator by depressing syringe plungers using VERY SLOW STEADY PRESSURE. To stop spray delivery, release pressure on syringe plungers while maintaining gas flow by holding foot switch down for 3-5 seconds after applying Tisseel to clear the Applicator's Replaceable Tip.
- If the Replaceable Tip becomes occluded during use, as indicated by no spray or by no movement of the ball in the Regulator flow meter, use the Tip Alignment Tool to remove the plugged tip by unscrewing it in a counterclockwise motion (Figure 6.1). Using a sterile cloth, wipe any clotted material or fluid from the exposed metal tube ends. Use the opposite chamber of the Tip Alignment Tool to install a new tip by screwing it in a clockwise motion (Figure 6.2). Ensure Replacement Tip is secured firmly to prevent accidental Tip detachment.
- To release the Applicator from the syringe Luers, press the release button on the Snap Lock as shown in Figure 4.
- Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

Baxter, DuploSpray and Tisseel are trademarks of Baxter International Inc., or its subsidiaries.

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Baxter, DuploSpray ja Tisseel ovat Baxter International Inc:n tai sen tytäryhtiöiden tavaramerkkejä.