
FibriJet®

ENDOSCOPIC APPLICATOR
WITH MIXING TIP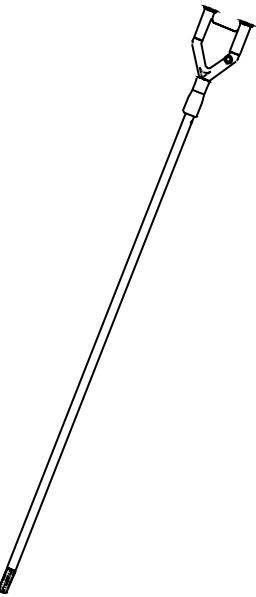

Instructions for Use
Mode d'emploi
Gebruiksaanwijzing
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Instruções de Utilização

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

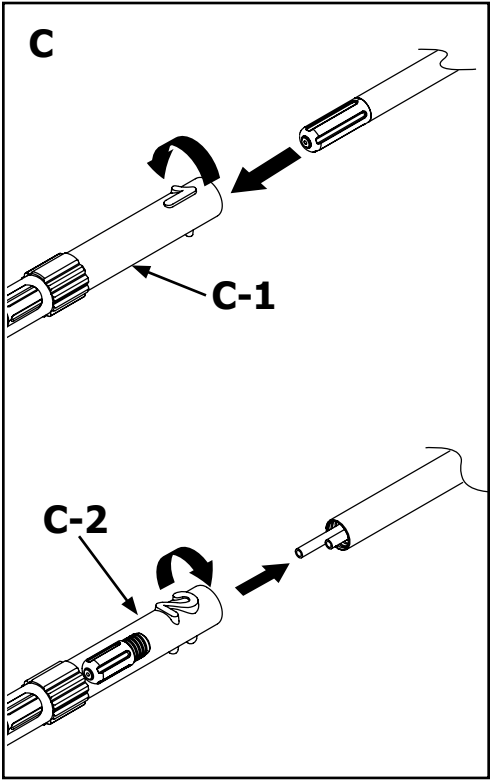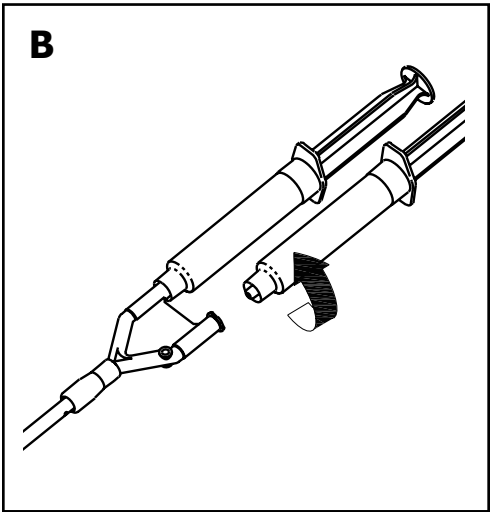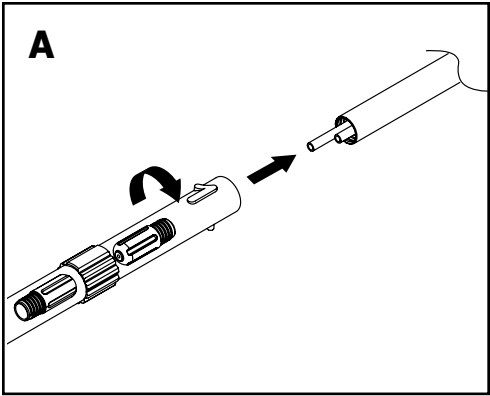
FibriJet Endoscopic Applicator With Mixing Tip

Before using, read the
following information:

****IMPORTANT NOTE****
This device is intended for use with a syringe
or compatible applicator with a standard luer
lock which is not included. This must be ordered
separately. Make sure you have both parts before
proceeding.

DESCRIPTION/ INDICATIONS FOR USE:

The Endoscopic Applicator with Mixing Tip is intended
for application of two non-homogenous liquids.

CONTRAINDICATIONS:

None known

PRECAUTIONS:

- If lever force is applied to the Endoscopic
Applicator, grasp the upper portion of the stainless
steel shaft. Do not use the syringe assembly as a
hold point.
- Ensure spray tip is secured to prevent accidental
tip detachment.
- Retain Tip Alignment Tool which holds a second
Replacement Tip provided for use if the first tip
becomes occluded.
- (Figure C, C-1, and C-2) If Replacement Tip
become occluded during use, (indicated by no
spray or no movement of the call in the gas flow
gage) use the Tip Alignment Tool (which the
installed Tip came from) to remove plugged Tip by
unscrewing counterclockwise.

INSTRUCTIONS FOR USE:

1. REMOVE the applicator from its sterile package
using sterile technique.
2. (Figure A) Use Tip Alignment Tool to attach the tip
to the Applicator.
3. (Figure B) assemble the filled syringes to the
Applicator and install the syringe holder and
plunger link per Applicator Assembly instructions.
4. Dispense liquids into Applicator by depressing
syringe plungers using steady pressure.
5. Dispose of as Biohazard waste. Device will degrade
if reprocessed. No effective cleaning process has
been developed to prevent cross contamination.
Contamination of a reprocessed device may lead to
injury, illness, or death of a patient.

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