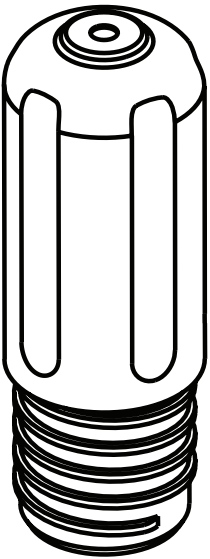
FibriJet®

Replacement Tip

Instructions for Use

Mode d'emploi

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Bruksanvisning

Manufactured in Mexico for:

Micromedics, Inc.
1270 Eagan Industrial Road
St. Paul MN 55121-1385, USA
651-452-1977
800-624-5662

CE 0297


EC REP

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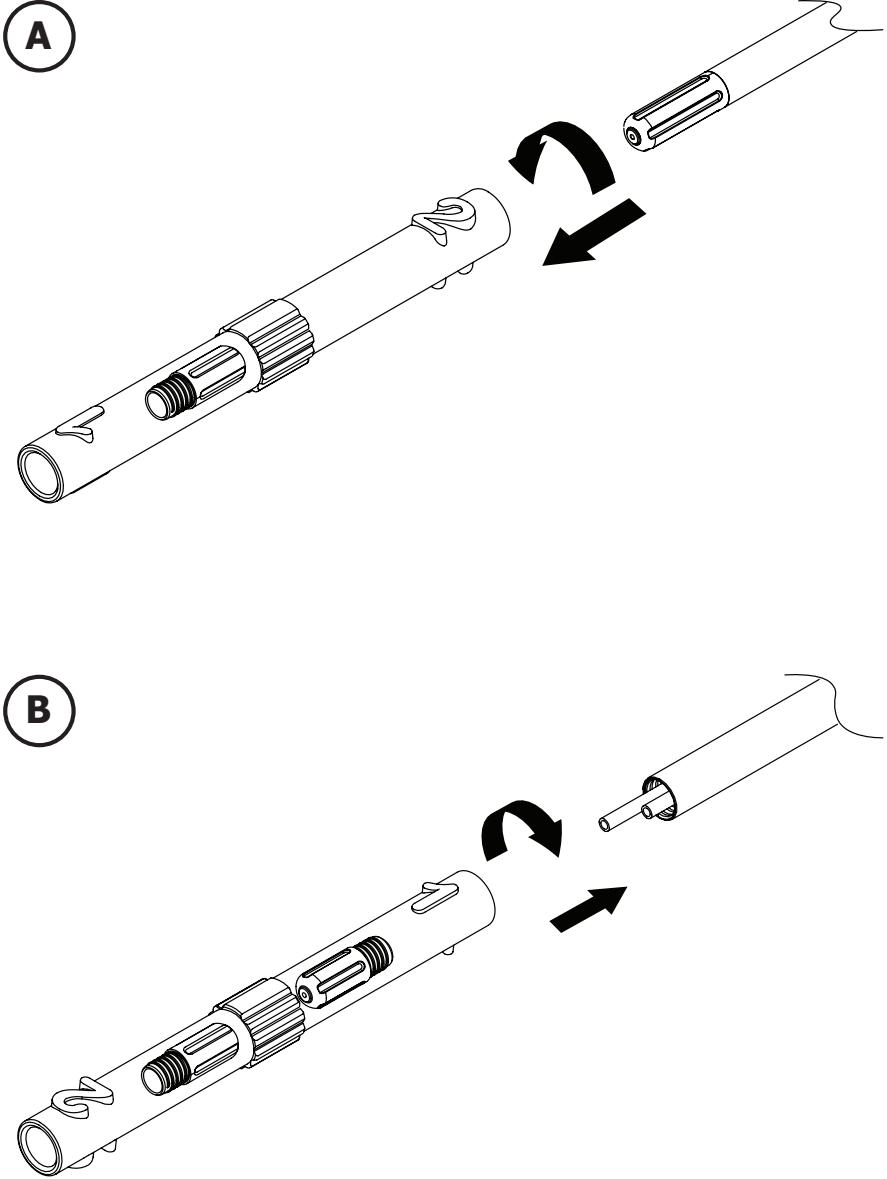FibriJet

Replacement Tip

 Before using, read the following information:

****IMPORTANT NOTE****
This device is intended for use with Micromedics products which are not included. The Endoscopic Applicator and Gas Assisted Endoscopic Applicator must be ordered separately. Make sure you have both parts before proceeding.

DESCRIPTION/ INDICATIONS FOR USE:
For replacement of the instrument tip on the Endoscopic Applicator and Gas Assisted Endoscopic Applicator.

CONTRAINDICATIONS:
None known

- PRECAUTIONS:**
- Ensure spray tip is secured to prevent accidental tip detachment.
 - Retain Tip Alignment Tool which holds a second Replacement Tip provided for use if the first tip becomes occluded.
 - (A) If Replacement Tip becomes occluded during use, (indicated by no spray or no movement of the ball in the gas flow gage) use the Tip Alignment Toll (which the installed Tip came from) to remove plugged Tip by unscrewing counterclockwise.

- INSTRUCTIONS FOR USE:**
- Wipe away any fluid from the ends of the exposed inner tubes.
 - REMOVE the spray tip from its sterile package using sterile technique.
 - (B) Use the Tip Alignment Tool to install the new Tip.
 - Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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