
DuploSpray MIS Applicator

Instructions for Use
Mode d'emploi
Gebrauchsanleitung
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Brugsanvisning
Οδηγίες χρήσης
Instruções de utilização
Bruksanvisning
Pokyny k použití
Instrukcja obsługi
Kullanım talimatları
Указания за употреба
Uputstva za uporabu
Käyttöohjeet

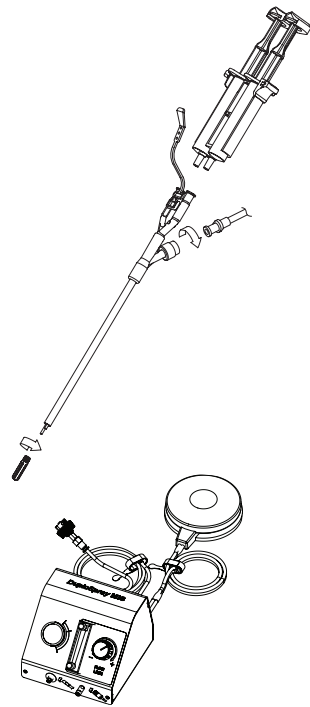CE 0297

EC REP

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DuploSpray MIS Applicator

Before using, read the following information:

INTENDED USE:

The DuploSpray MIS Applicator is intended for the application of TISSEEL/TISSUCOL Fibrin Sealant.

CONTRAINDICATIONS: NONE**WARNING:**

Connect to regulated CO₂ gas source only; maximum input pressure not to exceed 100 psi. To avoid possible gas embolism, do not directly spray into circulatory pathways. Any application of pressurized gas may be associated with a potential risk of gas embolus, tissue rupture or gas entrapment with compression, which may be life threatening. Be sure to take appropriate measures to address these risks by observing the recommended minimum spraying distance of 2cm (optimal working distance 3cm) and the maximal flow rate of 2.0 standard liters per minute.

INSTRUCTIONS FOR SETUP:

[Note: Regulator should be allowed to adjust to room temperature for two hours prior to use.]

- Set up the Regulator (Figure 1).

Attach gas supply hose to source of medical grade CO₂ gas. Place Regulator (Baxter REF 0600032) in proximity to operative field. Place foot switch on a flat surface in convenient location within reach.

Turn on gas supply and adjust gas source pressure to 100 psi/7 bar. The pressure gauge on faceplate of Regulator should indicate 18-22 psi/1.24-1.5 bar or in green zone as indicated by color graphic printed on faceplate (Figure 1.1).

Note: Pressure Gauge readings above 25 psi/ 1.7 bar (Red Zone) will cause Regulator's internal safety relief valve to release pressure safely inside Regulator assembly with an audible signal. Should this occur, immediately remove Regulator from service and call Customer

- Adjust Regulator flow (Figure 2).

While holding foot switch down, use gas flow control knob on front panel to adjust gas flow to 1.0 -2.0 liters per minute (Figure 2.1). Check gas flow by noting height of ball in flow gauge while holding foot switch down (Figure 2.2).

- Assemble Spray Set.

Use Tip Alignment Tool to thread sterile Replaceable Tip onto Applicator until seated against end of the Applicator Shaft (Figure 3). Retain Tip Alignment Tool which holds a second Replaceable Tip provided for use if first tip becomes occluded.

Attach gas supply line, clear luer connector, to sterile Applicator (Figure 4.1) and attach patient vent line, red luer connector, to available female luer on Trocar cannula vent valve (Figure 4.2) The twin-tube assembly can be easily separated as necessary to allow freedom of movement.

- Attach Spray Set to Regulator (Figure 5).

- Assemble filled DUPLOJECT Syringes to Applicator. Fasten pull strap to DUPLOJECT syringe
- Applicator is ready for use.

INSTRUCTIONS FOR USE:

- Press foot switch to start gas flow prior to applying TISSEEL/TISSUCOL.
- Check gas flow gauge on Regulator before inserting Applicator - if Flow Level Ball does not move when foot switch is depressed, Applicator Tip is occluded and should be replaced (See Step 4 below).
- While activating foot switch, dispense TISSEEL/ TISSUCOL through Applicator Tip by depressing syringe plungers using VERY SLOW STEADY PRESSURE. To stop spray delivery, release pressure on syringe plungers while maintaining gas flow by holding foot switch down for 3-5 seconds after applying liquids to clear Applicator Tip.

Flow Rate	1.0 slpm - 2.0 slpm		
Distance	2cm	3cm	5cm
		recommended	

- If Tip becomes occluded during use, as indicated by no spray or by no movement of ball in gas flow gauge, use Tip Alignment Tool to remove plugged Tip by unscrewing counterclockwise (Figure 6.1). Using a sterile cloth, wipe any clotted material from exposed tube ends. Use opposite chamber of the Tip Alignment Tool to install new tip and tighten securely with finger pressure (Figure 6.2).
- Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination which may lead to injury, illness, or death of a patient.

INFORMATION FOR PATIENT VENT LINE FEATURE:

The Patient Vent line attached to the Trocar cannula vent valve will only vent gas out when foot pedal is depressed on Regulator. A Trocar cannula with a vent valve must be used for this feature to work properly. After connection to Trocar cannula, ensure vent valve is fully open on Trocar cannula prior to use.

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