
RIGID ENDOSCOPIC TIP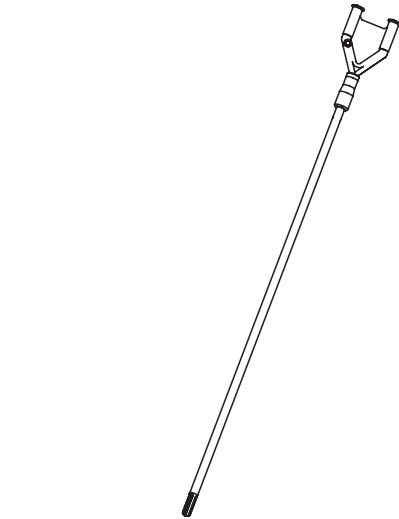

Instructions for Use

Mode d'emploi
Gebrauchsanweisung
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Instruções de Utilização

DIST

56 EAST BELL DRIVE
P.O. BOX 587
WARSAW, IN 46581 USA

Manufactured in Mexico for:

Micromedics, Inc.
1270 Eagan Industrial Road
St. Paul, MN 55121-1385 USA
651-452-1977

CE 0297




EC REP

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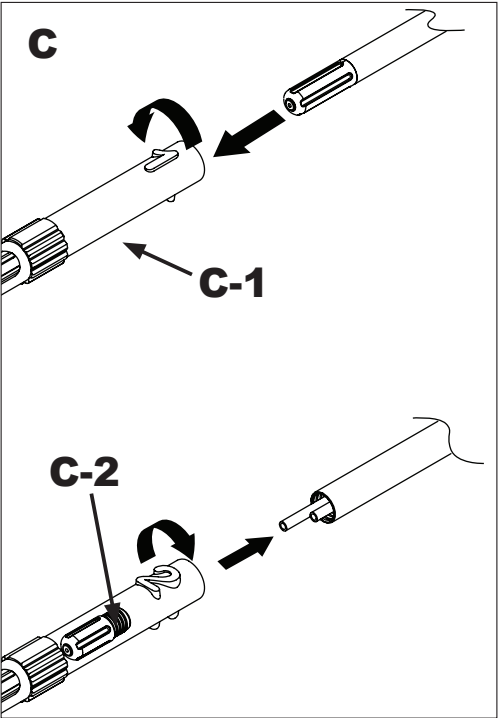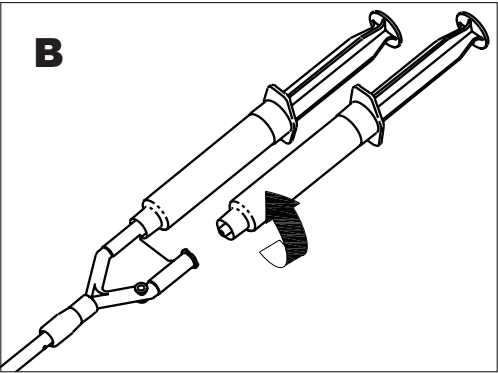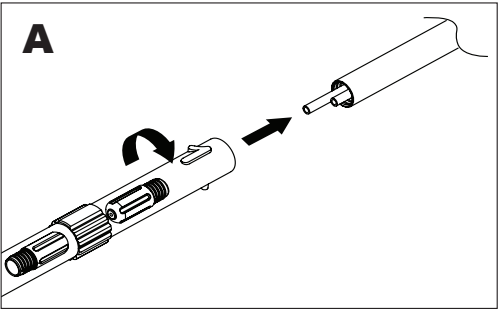RIGID ENDOSCOPIC TIP

Before using, read the following information:

****IMPORTANT NOTE****
This device is tended for use with a syringe or compatible applicator with a standard luer lock. This must be ordered separately. Make sure you have all parts before proceeding.

DESCRIPTION/INDICATIONS FOR USE

The Rigid Endoscopic Tip is intended for application of two non-homogeneous liquids.

CONTRAINDICATIONS

None Known

PRECAUTIONS

- If lever force is applied to the Endoscopic Applicator grasp the upper portion of the stainless steel shaft. Do not use the syringe assembly as a hold point.
- Ensure spray tip is secured to prevent accidental tip detachment.
- Retain Tip Alignment Tool which holds a second Replacement Tip provided for use if the first tip becomes occluded.
- (Figure C, C-1, and C-2) If Replacement Tip becomes occluded during use, (indicated by no spray or no movement of the ball in the gas flow gage) use the Tip Alignment Tool (which the installed Tip came from) to remove plugged Tip by unscrewing counterclockwise.

INSTRUCTIONS FOR USE

- REMOVE the applicator, tubing from its sterile package using sterile technique.
- (Figure A) Use Tip Alignment Tool to attach the tip to the Applicator
- (Figure B) Assemble the filled syringes to the Applicator and install the syringe holder and plunger link per Applicator Assembly instructions.
- Dispense liquids into Applicator by depressing syringe plungers using steady pressure.
- Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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