Instructions for Use
Mode d'emploi
Gebrauchsanweisung
Instrucciones de uso
Istruzioni per l'uso
Gebruiksaanwijzing
Bruksanvisning
Instruções de Utilização
Οδηγίες χρήσης

Manufactured in Mexico for:



Micromedics, Inc.
1270 Eagan Industrial Road
St. Paul MN 55121-1385, USA
651-452-1977
800-624-5662

CE 0297

EC REP
MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany

Symbol Reference Key Définition des symboles / Erklärung der Symbole / Definición de los símbolos / Legenda dei simboli / Verklaring van symbolen / Nøkket til symbolreferanser / Legenda para Referência dos Símbolos / Υπόμνημα αναφοράς συμβόλων		
SPRAY SET, 360° GAS ASSISTED ENDOSCOPIC APPLICATOR, DUAL CANNULA		
Spray Set, 360° Gas Assisted Endoscopic Applicator, Dual Cannula Ensemble de pulvérisation, applicateur endoscopique à activation gazeuse 360°, canule double Sprühset, 360° Gasgestützter endoskopischer Applikator - Doppelkanüle Conjunto pulverizador, aplicador endoscópico de 360° con cánula doble asistido por gas Set di spruzzo per applicatore endoscopico assistito a gas a 360° con cannula doppia Sproeisett, 360° gasaangedreven endoscopische applicator met dubbele canule Spraysett, 360° gassassistert endoskopisk applikator, dobbel kanyle Conjunto de nebulização, Aplicador endoscópico assistido por gás a 360°. Cânula dupla Σετ ψεκασμού, ενδοσκοπικό απλικατέρ ψεκασμού 360° με υποβοήθηση αερίου και διπλό σωληνίσκο		
Manufactured in Mexico for:		
Manufactured in Mexico for: Fabriqué au Mexique pour : Hergestellt in Mexiko für: Hecho en México para: Prodotto in Messico per:	Vervaardigd in Mexico voor: Produsert i Mexico for: Fabricado no México para: Κατασκευάζεται στο Μεξικό για την:	
	Not made with natural rubber latex Ne contient pas de latex Nicht aus Naturkautschuklatex gefertigt No está fabricado con látex de goma natural Non contiene lattice di gomma naturale Niet vervaardigd met natuurlijke rubberlatex Ikke produsert med lateks av naturgummi Não contém látex de borracha natural Δεν κατασκευάζεται με φυσικό ελαστικό λατέξ	
	Catalog Number Numéro de référence Katalognummer Número de catálogo Numero di catalogo	Catalogusnummer Katalognummer Número de catálogo Αριθμός καταλόγου
	Manufacturer Fabricant Hersteller Fabricante Produttore	Fabrikant Produsent Fabricante Κατασκευαστής
	Caution Attention Vorsicht Cuidado Attenzione	Let op Obs! Cuidado Προφύλαξη
	Consult Instructions for Use Consulter le mode d'emploi Gebrauchsanleitung konsultieren Consulte las Instrucciones de uso Consultare le Istruzioni per l'uso	Raadpleeg de gebruiksaanwijzing Se bruksanvisningen Consultar as Instruções de utilização Συμβουλευθείτε τις οδηγίες χρήσης
	Lot Number Numéro de lot Chargenbezeichnung Número de lote Numero di lotto	Partijnummer Partinummer Número de lote Αριθμός παρτίδας
	Use By Utiliser avant Verwendbar bis Fecha de caducidad Data di scadenza	Uiterste gebruiksdatum Brukes før Validade Χρήση έως
		Sterilized using EthyleneOxide Stérilisé à l'oxyde d'éthylène Mit Ethylenoxid sterilisiert Esterilizado por óxido de etileno Sterilizzato con ossido di etilene Gesteriliseerd met ethyleenoxide Sterilisert med etylenoksid Esterilizado por óxido de etileno Αποστειρωμένο με οξείδιο του αιθυλενίου

	Authorized Representative in the European Community Représentant agréé dans l'Union européenne Autorisierte EU-Vertretung Representante autorizado en la Comunidad Europea Rappresentante autorizzato per l'Unione Europea Erkend vertegenwoordiger in de Europese Gemeenschap Autorisert representant i EU Representante autorizado na Comunidade Europeia Εξουσιοδοτημένος αντιπρόσωπος στην Ευρωπαϊκή Κοινότητα	
	Do not use if the packaging is damaged Ne pas utiliser si le conditionnement est endommagé Bei beschädigter Verpackung nicht verwenden No debe utilizarse si el envase está dañado Non utilizzare se la confezione è danneggiata Niet gebruiken als verpakking beschadigd is Skal ikke brukes hvis emballasjen er skadet Não utilizar se a embalagem estiver danificada Μη χρησιμοποιείτε αν η συσκευασία έχει υποστεί ζημιά	
	Do Not Reuse Ne pas réutiliser Nicht wiederverwenden No reutilizar Non riutilizzare	Niet opnieuw gebruiken Skal ikke gjenbrukes Não reutilizar Μην επαναχρησιμοποιείτε
	Do not resterilize Ne pas restériliser Nicht resterilisieren No lo vuelva a esterilizar Non ristilizzare	Niet hersteriliseren Skal ikke steriliseres på nytt Não reesterilizar Μην επαναπαστειώνετε

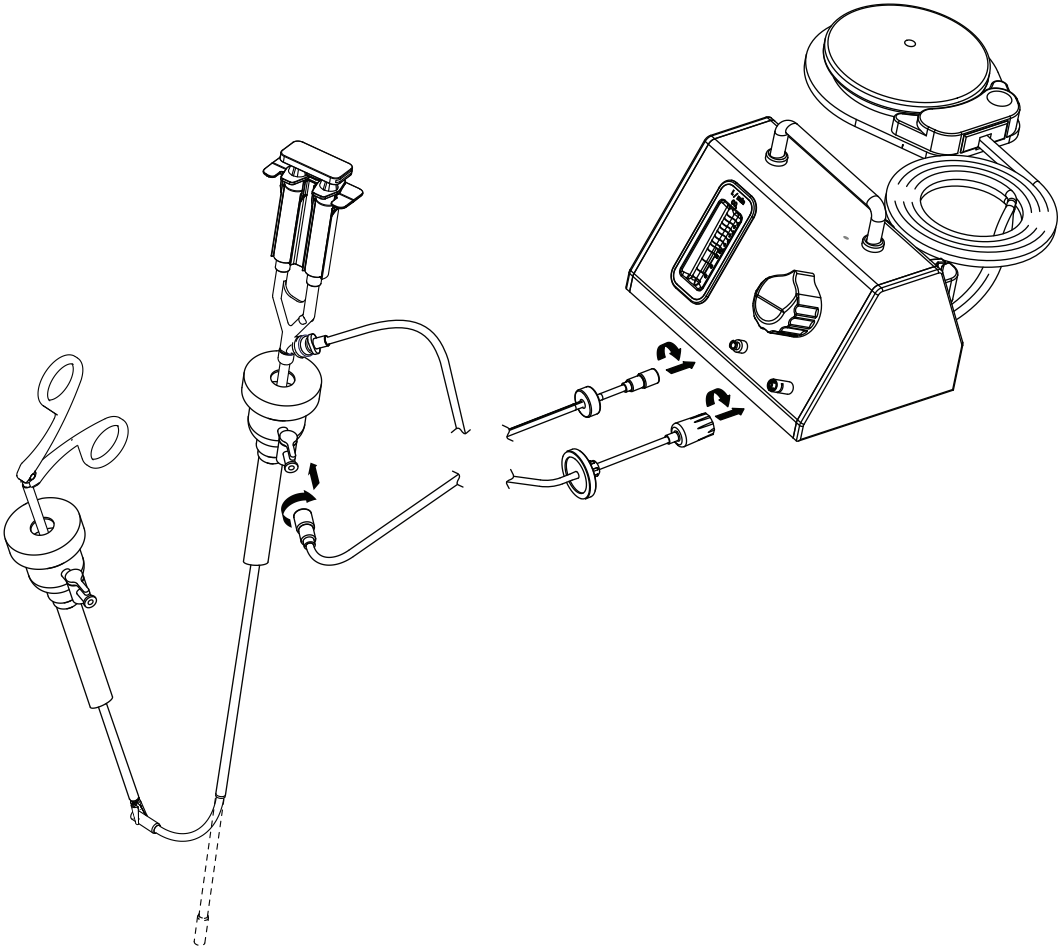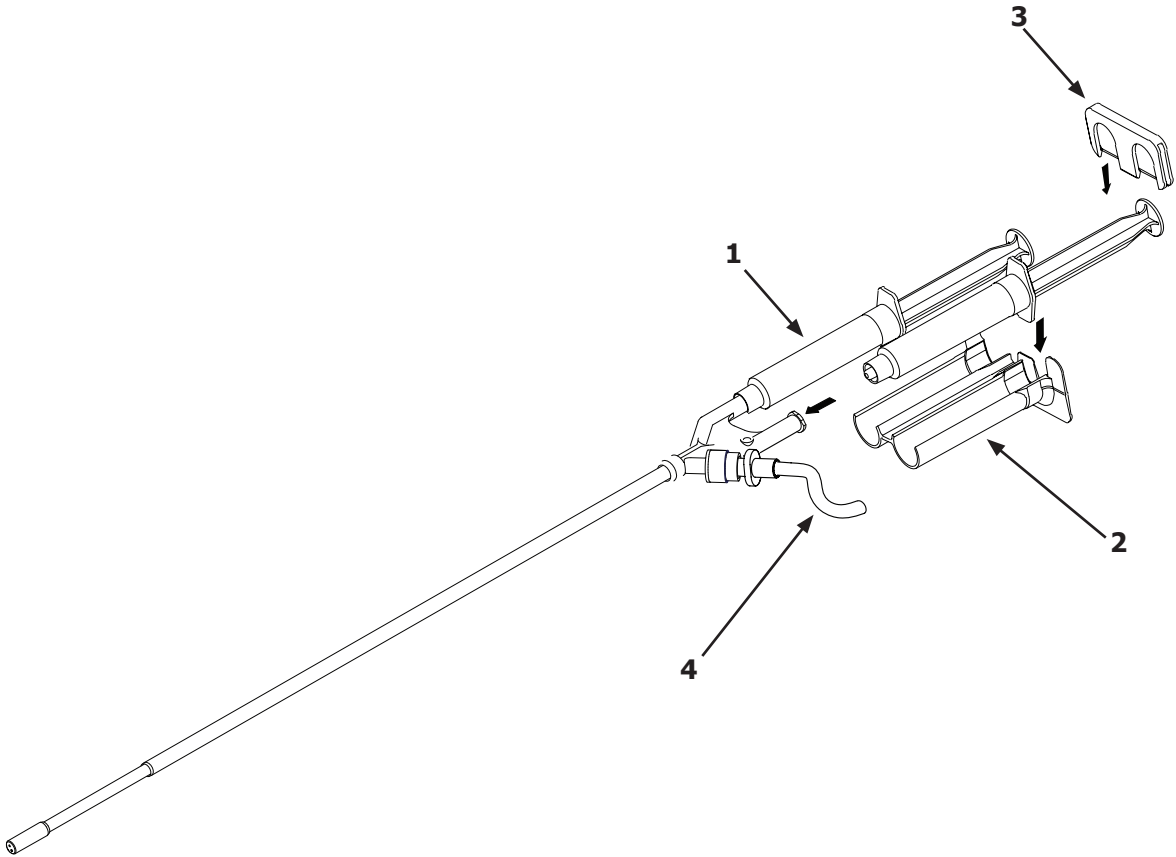EN

Spray Set, 360° Gas Assisted Endoscopic Applicator, Dual Cannula

!

Before using, read the following information:

****IMPORTANT NOTE****
This device must be connected to a Regulator (such as SA-6030) and is only to be used with Nitrogen, CO2, or hospital air. Regulator and Applicator Assembly are not included. They must be ordered separately. Make sure you have both parts before proceeding.

DESCRIPTION/ INDICATIONS FOR USE:

The Gas Assisted Endoscopic Spray Applicator is intended for the application for two homogeneous liquids.

CONTRAINDICATIONS:

None known

PRECAUTIONS:

- When using pressurized spray devices, changes in blood pressure, pulse, oxygen saturation, and end tidal CO2 should be monitored because of the possibility of occurrence of gas embolism.
- Connect to regulated CO2 gas source; maximum input pressure not to exceed 100 psi (7 bar). To avoid possible gas embolism, do not directly spray into circulatory pathways. For use by or on the order of a licensed physician.

APPLICATOR ASSEMBLY:

- Refer to the FibriJet® Regulator instructions for use for assembly and set-up.
- Assemble endoscopic applicator per pictorial.

INSTRUCTIONS FOR USE

- Press the foot switch to start gas flow prior to applying liquids. (Gas flow may be used to dry operative site.)
- Check gas flow gauge on Regulator before inserting Applicator. Set gas flow on Regulator before inserting Applicator. Set gas flow rate to 4.0 ± 0.5 L/min with Applicator attached.
- While activating foot switch, dispense liquids into the Applicator Tip by depressing syringe plungers using VERY SLOW STEADY PRESSURE. Recommended Applicator distance from spray site should be approximately 3-5 cm. To stop spray delivery, release pressure on syringe plungers while maintaining gas flow by holding foot switch down for 3-5 seconds after applying liquids to clear Applicator Tip.
- Applicator Tip may be angled by grasping and turning with an endoscopic grasper at the distal end.
- Dispose of as Biohazard waste. Device will degrade if reprocessed. No effective cleaning process has been developed to prevent cross contamination. Contamination of a reprocessed device may lead to injury, illness, or death of a patient.

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